EQUIA Forte® HT

GC America www.gcamerica.com

dentaladvisor.com

RATING SYSTEM: Excellent + + + + + Very Good + + + + Good + + +





Key features: Glass hybrid restorative material • Bulk fill • Requires the use of a triturator • System includes light-cured **EQUIA Forte® Coat**

Description

EQUIA Forte® HT:

- · New innovative member of GC's Glass Hybrid family
- Bulk-fill glass hybrid restorative system
- Designed to provide a unique long-term solution for your posterior restorations
- · Ideal restorative treatment for high-risk caries and pediatric patients
- Intelligent particle size and distribution allowing superb handling
- Optimal marginal seal for long-term resistance to microleakage
- · High, rechargeable, fluoride release at tooth-restorative interface
- · System that includes resin coating: EQUIA Forte Coat

Indications

- Class I restorations
- · Stress-bearing Class II restorations (with specific preparation)
- Non-stress bearing Class II restorations
- Intermediate restorations
- Class V and root surface restorations
- Core build ups

Unique Attributes

- Improved translucency and strength
- System includes *EQUIA Forte Coat*, resin coating, which increases strength
- New flip top dispensing for EQUIA Forte Coat



Clinical Tips

- This material is soft and cuts really easy when prepping for a core buildup.
- You want to sculpt it before it hardens too much as it gets sticker as it sets up.
- Use a fine diamond to finish.

than using an instrument.

- For Class V restorations, packing the material with your finger works well to contour to the root structure and is sometimes easier
 "SET UP QUICKLY
- Can use with cavity conditioner to improve adhesion.

"SET UP QUICKLY AND DID NOT STICK TO MY MATRIX BAND."

Improve danesion.

Evaluators' Comments

"Nice finish and polish with the EQUIA Forte Coat."

"Easy to pack, even in subgingival areas."

"When doing a core buildup and having to pack and contour to create a wall, this material makes it easy while also being less messy."

"Placed it on a sensitive tooth and it took the sensitivity away."

"I don't like that it has to be triturated."

"Great for pediatric patients, however, once you consider the activating/dispensing gun as well, it can be bulky."

"A lot of steps, especially if you use the cavity conditioner."

