Prior to use, carefully read the instructions for use.

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ES

# GC Fuji™ CAPSULE

## RADIOPAQUE GLASS IONOMER LUTING CEMENT IN CAPSULES

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

1. Cementation of metal-based inlays, onlays, crowns, bridges and posts
2. Cementation of high strength (zirconia based) all ceramic crowns and bridges

### CONTRAINDICATIONS

1. Pulp capping.
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

|   |           |
|---|-----------|
| Powder / Liquid ratio (g/g)                                     | 0.33/0.18 |
| Mixing time (sec.)  | 10"       |
| Working time (min., sec.) (at 23 °C) (from start of mixing)     | 2'15"     |
| Final finishing commencing time (after seating the restoration) | 4'30"     |

#### 1. TOOTH PREPARATION




- a) Prepare tooth in usual manner. For pulp capping, use calcium hydroxide.
- b) Clean the prepared teeth with pumice and water.
- c) Rinse thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with an oil-free air. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).

#### 2. RESTORATION PREPARATION

Make sure that the restoration is pretreated and handled according to the manufacturer's instructions.

#### 3. CAPSULE ACTIVATION AND MIXING

- a) Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 1).
- b) To activate the capsule, push the plunger until it is flush with the main body and hold it down for 2 seconds (Fig. 2).  
Note:
  - i) Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.
  - ii) The capsule should be activated just before mixing and used immediately.
- c) Immediately remove the capsule and set it into a mixer (or an amalgamator) and mix for 10 seconds (~4,000 RPM) (Fig. 3).  
Note: When using a rotational mixing device, it is recommended not to use the centrifuge option.

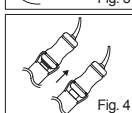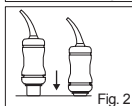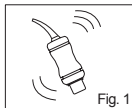#### 4. DISPENSING

- a) Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
- b) Make two clicks to prime the capsule (Fig. 4). To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
- c) Extrude directly into the restoration.
- d) To remove the used capsule, push the applicator release button. Twist the capsule and pull upwards.

#### 5. CEMENTATION

- a) Coat the internal surface of the restoration with sufficient cement and seat immediately. The working time is 2 minutes 15 seconds from start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
- b) Maintain moderate pressure.
- c) Start removing excess cement when the excess cement feels rubbery.
- d) Finishing can be started 4 minutes 30 seconds after seating the restoration.

### STORAGE

Recommended for optimal performance. Store the capsules in the original aluminum foil in a cool and dark place. (4-25°C) (39.2-77.0°F).

### SHADE

Light yellow.

### PACKAGE

48 capsules

Average contents per capsule : 0.33g powder and 0.18g (0.14mL) liquid.

Single Shade Export package: 200 capsules

Minimum net volume of mixed cement per capsule : 0.19mL.

Option: GC CAPSULE APPLIER (1 piece).

### CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. This product is not indicated for filling or core build-up.
4. Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available for the Americas at:

<http://www.gcamerica.com>

They can also be obtained from your supplier.

Last revised: 11/2019

Rx Only

MANUFACTURED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by  
GC AMERICA INC.  
3123 West 127th Street, Alsip, IL 60803 U.S.A.  
[www.gcamerica.com](http://www.gcamerica.com)  
TEL: +1-708-597-0900

PRINTED IN THE U.S.A.

RESPONSIBLE MANUFACTURER IN CANADA  
GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A

GC SOUTH AMERICA  
Rua Heliadora, 399, Santana - São Paulo, SP - Brasil  
CEP: 02022-051 - TEL: +55-11-2925-0965  
CNPJ: 08.279.999/0001-61  
RESP. TEC. Miyara De Santos Ribeiro - CRO/SP: 105.982

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