# MI Varnish™

Prior to use, carefully read  
the instructions for use.

EN

Topical Fluoride Varnish with Calcium and Phosphate

For use only by a dental professional in the recommended indications.

## DESCRIPTION

A fluoride containing varnish to treat hypersensitive teeth and to aid in the prevention of cavities.

## MEDICINAL INGREDIENTS

Sodium fluoride 5% wt/vol, CPP-ACP (Casein Phosphopeptide Amorphous Calcium Phosphate) 2% wt/vol

## NON-MEDICINAL INGREDIENTS

Polyvinyl acetate, Ethanol, Ethylene Glycol Monoethyl Ether, Hydrogenated Rosin, Silicon Dioxide, Flavor, Sucralose

## RECOMMENDED DOSE

Adults and children age 1 year and older. MI Varnish can be applied every 3-6 months. MI Varnish should remain undisturbed on the teeth for 4 hours.

## CONTRAINDICATIONS

1. Do not use on patients with ulcerative gingivitis and stomatitis.
2. Casein Phosphopeptides are derived from milk casein. Do not use this material on patients with a proven or suspected milk protein allergy.

## PRECAUTIONS

1. In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
2. When MI Varnish is applied, patients should not use other prescriptive fluoride treatment such as fluoride gels during the same day. The routine use of fluoride tablets by patients should be interrupted for several days after treatment.

## DIRECTIONS FOR USE

1. Tooth surfaces should be cleaned and dried before application of MI Varnish. A dental prophylaxis is not required.
2. Peel off the foil lid of the unit dose container of MI Varnish.
3. Apply a thin, uniform layer of MI Varnish on teeth/surfaces using a disposable brush. Note: If separation is observed, stir with the disposable brush before application.
4. MI Varnish sets when in contact with water or saliva.
5. MI Varnish should remain undisturbed on the teeth for four hours. Instruct patients to avoid hard, hot or sticky foods, tooth brushing and flossing, products containing alcohol (oral rinses, beverages, etc.) during this time period.

## ADVERSE REACTIONS

Edematous swelling has been reported only in rare instances in some fluoride varnish products, especially after application to extensive surfaces. Although extremely rare, dyspnea (shortness of breath) has occurred in asthmatic children. Nausea has been reported when extensive applications have been made to patients with sensitive digestive systems.

If required, varnish film can be removed by thoroughly brushing once set.

## FLAVOR

Fresh Strawberry

Fresh Mint

## STORAGE

Recommended for optimal performance, store in a cool, dry and low humidity area away from direct sunlight (8-25°C) (46.4-77.0°F).

## PACKAGE

One unit dose package contains 0.44g/0.4mL

Unit dose packages, Qty: 50

Disposable brushes, Qty: 50

Note: 1mL of MI Varnish contains 50mg of sodium fluoride (22.6mg fluoride ion).

## CAUTION

Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.


1. MI Varnish is flammable. Do not use near open flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from direct sunlight.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. Avoid swallowing or digestion of material.
4. MI Varnish unit dose and brush are disposable. Do not reuse.
5. In case of contact with clothes, table or floor, immediately wipe off with alcohol.
6. Do not mix with other products.
7. Dispose of all waste according to local regulations.
8. MI Paste Plus® should not be used on the same day but can be used from the following day.

NOTE  
RECALDENT™ (CPP-ACP) Technology was developed by School of Dental Science, The University of Melbourne, Australia.  
\*RECALDENT is a trademark used under license.  
RECALDENT™ (CPP-ACP) Technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available for the Americas at: <http://www.gcamerica.com>

They can also be obtained by your supplier.

Last revised: 07/2019

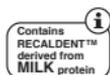Imported by:  
Radiant Global Logistics (Canada) Inc.  
3520 Laird Road, Unit 4  
Mississauga, ON L5L 5Z7  
NPN 80062417

**GC**

MANUFACTURED BY  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY  
GC AMERICA INC.  
3737 W. 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900  
[www.gcamerica.com](http://www.gcamerica.com)

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