


Instructions


For Use of Acri-Dense™ VI Pneumatic Curing Unit (5.6 Quarts)

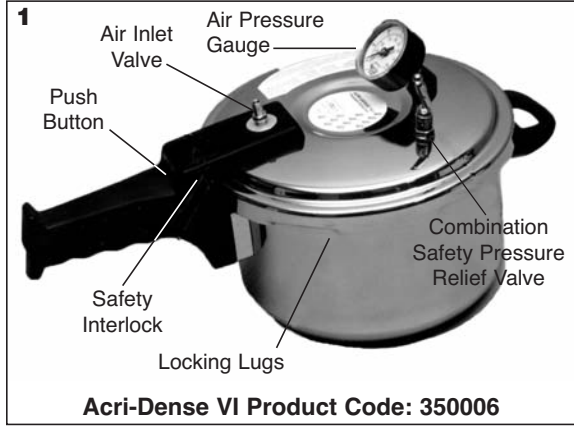



Caution: Do not use on stove or apply heat to unit. Do not open with force. Do not depress push button or attempt to open while unit is pressurized. Always exhaust pressure completely before attempting to open. If Unit contains pressure, the safety panel interlock (6 o'clock position on cover) prevents opening.




Open Combination Safety/Pressure Valve to vent all pressure from unit. Open unit (toggle straight up).

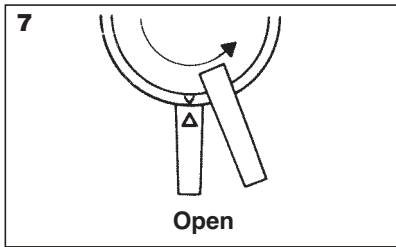

Open unit with toggle straight up and pressing down on the Push Button.

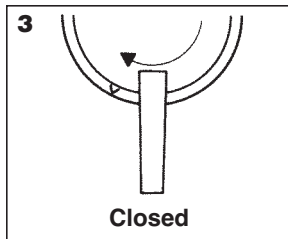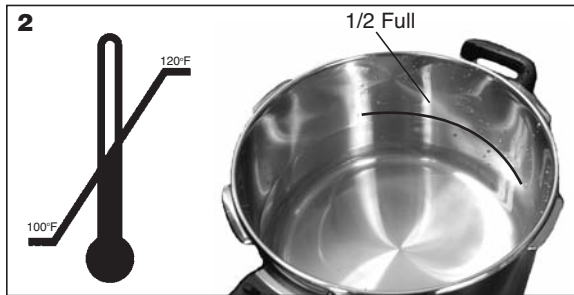

Pressure Unit has "Locking Lugs" on cover and pot. When closing bring cover "Lugs" directly over pot "Lugs".

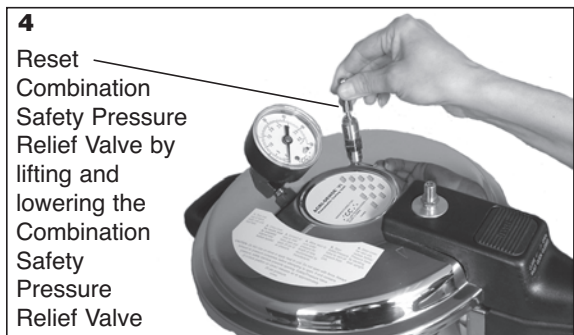

SAFETY:

- CAUTION: ALWAYS RELEASE AIR PRESSURE WHEN WORKING ON POT
- A spring-loaded Combination Safety/Pressure Relief Valve (2 o'clock position on cover) set to release air when pressure within unit exceeds 30 psi + 2 psi. It will then reseat itself between 23 and 24 psi.

TROUBLE-SHOOTING:

TROUBLE	SOLUTION
Loss of Air Pressure	<ul style="list-style-type: none"> • If leakage is around the lid edge... <ul style="list-style-type: none"> • Remove lid and rotate gasket 90° then pressurize. • Turn lid over and make sure the gasket placement is in front of the black pin near the handle. • If leaking from Air Pressure Gauge contact GC America Inc. • Release air pressure, remove lid and reclose lid while applying slight pressure directly in center of the lid.

MAINTENANCE:

For long service, lubricate the gasket with very light layer of petroleum jelly to prevent from drying out. If unit is opened and closed often, lubricate the underside of the pot rim and the gaskets well. DO NOT OVER STRETCH.

REPLACEMENT PARTS:

- | | |
|--|--|
| 350021 Air Inlet Valve | 350351 Combination Safety/Pressure Valve |
| 350041 Hose with Fitting (Optional) | 350491 Replacement Gasket for Acri-Dense VI |
| 350311 Air Pressure Gauge | |

MANUFACTURED FOR:



GC AMERICA INC.
ALSIP, ILLINOIS 60803
TEL (708) 597-0900
FAX (708) 371-5103
www.gcamerica.com

Order from your dealer