
COMPONENTS / TEILE / COMPOSANTS / COMPONENTI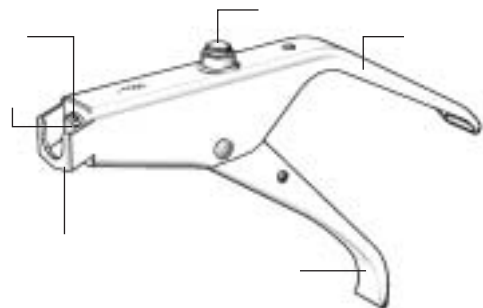

- ① Capsule Cover (polyacetal) / Kapselabdeckung (polyacetal) / Capsule cover (polyacetal) / Tappo capsula (poliacetale)
- ② Plunger (stainless steel) / Kolben (rostfreier Edelstahl) / Piston poussoir (acier inoxydable) / Stantuffo (acciaio inox)
- ③ Capsule holder (stainless steel) / Kapselhalter (rostfreier Edelstahl) / Mise en place de la capsule (acier inoxydable) / Portacapsula (acciaio inox)
- ④ Release button (polyacetal) / Löseknopf (polyacetal) / Bouton de remise à zéro du piston (polyacetal) / Pulsante di sgancio (poliacetale)
- ⑤ Grip (stainless steel) / Griff (rostfreier Edelstahl) / Grip (acier inoxydable) / Manico (acciaio inox)
- ⑥ Lever (stainless steel) / Betätigungshebel (rostfreier Edelstahl) / Poignée, gâchette (acier inoxydable) / Leva (acciaio inox)

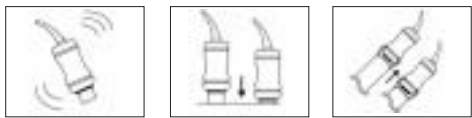
Fig. 1

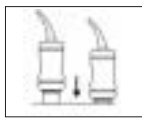
Fig. 2

Fig. 3

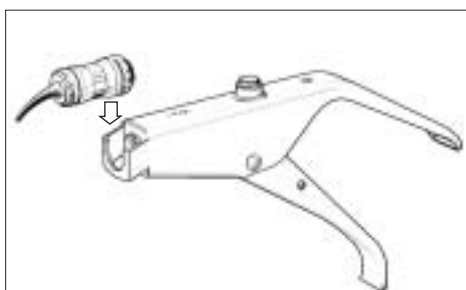
Fig. 4
Insert and fix securely. / Einsetzen und sichern. / Insérer et fixer correctement. / Inserire e fissare bene.

Fig. 5
Side view / seitlich / Vue de côté / Vista laterale

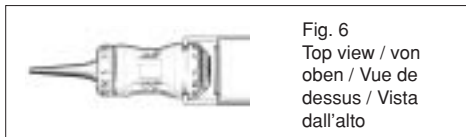
Fig. 6
Top view / von oben / Vue de dessus / Vista dall'alto

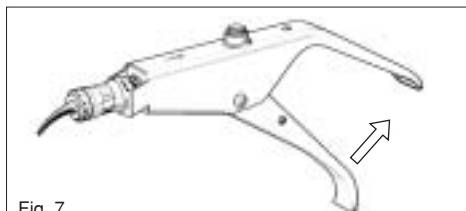
Fig. 7
2 priming strokes + 2 slow extruding strokes.
2 mal zum Vorbereiten, dann 2 mal langsam zum Applizieren betätigen.
2 premières pressions + 2 pressions "douces" pour l'extrusion
2 pressioni di innesco + 2 pressioni lente di erogazione

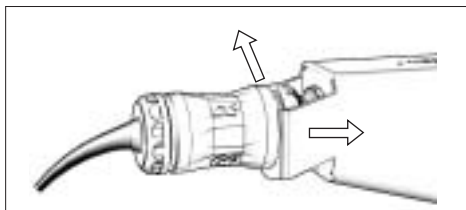
Fig. 8
① Pull the capsule. / Kapsel entnehmen. / Retirer la capsule. / Tirare la capsula.
② Retract the plunger. / Kolben entlasten. / Rétracter le piston. / Arretrare lo stantuffo.

Capsule Applier IV

For use only by a dental professional in the recommended indications.

USE

Activation and dispensing of encapsulated GC cements.

DIRECTIONS FOR USE

1. Capsule activation
Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 1). Push the plunger of the capsule until it's flush with the main body (Fig. 2), place into the capsule applicator and click the lever once (Fig. 3). The capsule is now activated.
2. Dispensing of mixed cement
a) After mixing in a mixer or amalgamator, insert and fix the capsule securely in the capsule holder (Fig. 4, 5, 6).

NOTE:

The capsule cannot be inserted if the plunger is extended. Retract the plunger by pressing the release button.

- b) Make 2 strokes with lever to prime the capsule. Make the third stroke slowly to check if the cement is extruded in the nozzle. Direct the capsule nozzle into the restoration or prepared tooth cavity and make slowly the fourth stroke to allow smooth dispensing of the cement (Fig. 7). Avoid putting too much force onto the applicator.

NOTE:

- 1) To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
- 2) In case of difficulty in extrusion, remove the capsule and check the applicator. If the applicator is in good condition, replace with a new capsule.

3. Removal of the capsule
Press the release button to retract the plunger. Remove the used capsule from the applicator by twisting and pulling upwards (Fig. 8).

CLEANING / DISINFECTING

Remove any remaining cement immediately from the applicator. Clean the applicator with a gauze soaked in alcohol or in an ultrasonic bath. In case of using chemical disinfectant / sterilizing agent, follow the manufacturer's instructions carefully.

NOTE:

When immersing the applicator in the chemical disinfectant for a prolonged period of time, it is recommended to use a disinfectant solution which contains rust inhibitor.

STERILIZATION

The applicator can be autoclaved at 121-134°C (250-273°F) for 10 to 30 minutes. Please follow manufacturer's instructions.

NOTE:

The operating (sterilizing and drying) temperature should not exceed 134°C (273°F), as this may cause deformation of the polyacetal parts.

CAUTION

1. Prior to use, sterilize the applicator.
2. Take care not to get your hand or fingers caught between the lever and the grip.
3. In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

STORAGE

Keep GC Capsule Applier IV in a dry place away from high moisture to prevent from rusting.

PACKAGE

Capsule Applier IV - 1 piece

Prior to use, carefully read the instructions for use. **GB**

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.
Yukadenshi Company Limited