Prior to use, carefully read the instructions for use

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P

## COE LUBRICANT

SEPARATING AGENT AND TISSUE BARRIER FOR CHAIRSIDE DENTURE RELINING.

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS:

COE LUBRICANT is used in conjunction with GC relene materials, COE-SOFT, COE-COMFORT, KOOLINER, GC RELINE HARD, and COE-RECT. It has two purposes: as a bonding barrier to prevent relene materials from bonding to the denture in areas where a bond is not desired and to insulate mucosal tissue and minimize sensation of heat characteristic of self curing exothermic reactions. The intended applications will vary with product type. Please see the instructions for the relene material used.

### CONTRAINDICATIONS:

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

*NOTE: Refer to the instructions for the specific soft denture liner to be used for possible contraindications.*

### DIRECTIONS FOR USE:

#### To use as a bonding barrier:

Areas of the denture that should not be bonded to new relene material should be painted with a thin coat of COE LUBRICANT immediately prior to seating the relene.

Assure that COE LUBRICANT is not applied to areas where a bond is desired, as it will reduce the bond of relene to the denture.

If COE LUBRICANT is improperly placed on the denture, assure complete removal before proceeding with the relene. If necessary, roughen the affected area.

After relining is complete, remove the denture from the mouth and wash COE LUBRICANT off with soap and water.

#### To use on mucosal tissue:

Immediately prior to placing dentures for relene, coat mucosal tissue with a thin layer of COE LUBRICANT. Avoid excessive use of COE LUBRICANT to assure complete bond of relene material to the denture.

After relining is complete, remove the denture from the mouth and wash COE LUBRICANT off with soap and water. Ask the patient to rinse away any residue with water.

### STORAGE:

Store at normal temperature 20-25°C (68 - 77°F)  
(Shelf life: 10 years from date of manufacture)

### PACKAGE:

COE LUBRICANT 5/8 oz. bottle

### CAUTION:

- In case of contact with eyes, flush immediately with water. If irritation develops, seek medical attention.
- In case of contact with skin, wash with soap and water. If irritation develops, seek medical attention.

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Alsip, Illinois 60803 U.S.A