Prior to use, carefully read the instructions for use.

EN

DENTIN CONDITIONER

POLYACRYLIC ACID CAVITY CONDITIONER

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

DENTIN CONDITIONER is a mild polyacrylic acid solution designed to remove the dentinal smear layer and to condition dentine, thus enhancing the bond between glass ionomer cement and the dentine.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

- After tooth preparation, apply DENTIN CONDITIONER to the bonding surfaces for 20 seconds using a cotton pellet or sponge.
- Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe. DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening).
- Apply GC Fuji BOND LC, GC Fuji II LC or other glass ionomer as a bonding agent, base/liner or restorative material.
- Close the bottle immediately after use.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life : 3 years from date of manufacture).

PACKAGE

Bottle of 25g (23.8mL).

CAUTION

- In case of contact with oral tissue or skin, flush immediately with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

Last revised : 05/2010

CE 0086

'GC.'

MANUFACTURED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL: +65 6546 7588

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