

# EXAIMPLANT ADHESIVE

Prior to use, carefully read  
the instructions for use.

---

For use only by a dental professional in the recommended indications.

## RECOMMENDED INDICATIONS

For adhering EXAIMPLANT to custom resin impression trays.

## CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

---

The logo for GC, consisting of the letters 'GC' in a bold, italicized, sans-serif font, with a diagonal slash through the 'G' and another diagonal slash after the 'C'.

## HOW TO USE

---

1. Relieve one layer of the inner surface of the custom (resin) tray using a carbide bur. Clean and dry. Using the brush or a cotton pellet, apply EXAIMPLANT ADHESIVE uniformly to the inner surface of the tray.
- 
2. Blow dry thoroughly with an air syringe. Load EXAIMPLANT into the tray.

## STORAGE

---

Store in a dry, cool place.

(Shelf life : 2 years from date of manufacture).

## PACKAGE

---

Bottle of 8g (10mL) with 2 brushes.

## CAUTION

---

1. In case of contact with skin or oral tissue, immediately remove with a sponge or cotton soaked in alcohol. Flush with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. The material is flammable and volatile. Do not use near naked flame. Keep away from sources of ignition. Keep away from sunlight. Use in a well ventilated place.



MANUFACTURED by  
**GC DENTAL PRODUCTS CORP.**  
2-285, Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by  
**GC CORPORATION**  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

**GC EUROPE N.V.**  
Research-Park, Interleuvenlaan 13, B-3001 Leuven, Belgium

**GC AMERICA INC.**  
3737 West 127th Street, Alsip, IL 60803 U.S.A.

**GC ASIA DENTAL PTE. LTD.**  
19 Loyang Way, #06-27 Singapore 508724

PRINTED IN JAPAN

13002096  
300931DK