GC Gold Label High Strength Posterior Restorative

RADIOPAQUE POSTERIOR GLASS IONOMER RESTORATIVE CEMENT

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Class I and II restorations in deciduous teeth.
- Non-load bearing Class I and Class II restorations in permanent teeth.
- Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
- Class V and root surface restorations.
- Core build-up.

CONTRAINDICATIONS

- Pulp capping.
- Avoid use of this product in patients with known allergies to glass ionomer cement.

DIRECTIONS FOR USE

Powder / Liquid Ratio (g/g)	3.6 / 1.0
Mixing Time (sec.)	25"-30"
Working Time (min., sec.)	2'00"
Net Setting Time (min., sec.)	2'20"

Test conditions : Temperature (23+/-1°C), Relative humidity (50+/-10%)

1. POWDER AND LIQUID DISPENSING

- The standard powder to liquid ratio is 3.6g/1.0g (1 level scoop of powder to 1 drop of liquid).
- For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- Hold the liquid bottle vertically and squeeze gently.
- Close bottles tightly immediately after use.






2. MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts.

Mix the first portion with all the liquid for 10 seconds. Incorporate the remaining portion and mix the whole thoroughly for 15-20 seconds (Fig. 1).

3. RESTORATIVE TECHNIQUE

- Prepare the tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping, use calcium hydroxide.
 - For optimal chemical bonding to the teeth, the smear layer produced in the preparatory step should be removed by applying either CAVITY CONDITIONER (10 seconds) or DENTIN CONDITIONER (20 seconds). For use in the Atraumatic Restorative Technique (ART), GC Gold Label High Strength Posterior Liquid should be applied for 15 - 20 seconds to the prepared surfaces using a cotton pellet or sponge.
 - Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe. DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening) (Fig. 2).
 - Mix the required amount of cement. Working time is 2 minutes from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
 - Transfer cement to the preparation using a syringe or other suitable instrument. Avoid air bubbles.
 - Form the preliminary contour and cover with a matrix if required (Fig. 3).
 - When set, immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) (Fig. 4).
4. FINISHING
- Final finishing under water spray using standard techniques can begin at 6 minutes after starting the mix (Fig. 5).
 - Apply a final coat of GC Fuji VARNISH or GC Fuji COAT LC (Fig. 6).
 - Instruct the patient not to apply pressure for 1 hour.

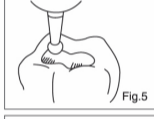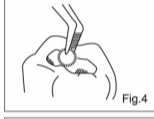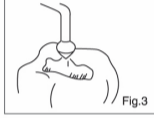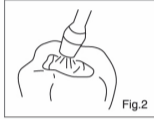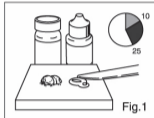STORAGE

Recommended for optimal performance, store at temperature of 4-25°C / 39.2-77.0°F. (Shelf life : 3 years from date of manufacture)

SHADES

A2, A3, A3.5, B2, B3, C4

Shade numbers according to Vita® shade guide.

© Vita is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGES

- 1-1 Package: 15g powder, 13.1g (10.5mL) liquid, powder scoop, mixing pad (No. 22)
 - Bottle of 15g powder with scoop.
 - Bottle of 13.1g (10.5mL) liquid.
 - Mini pack*: 5g powder, 5g (4.0mL) liquid, powder scoop
- * Mini pack is only available in limited shades (A2, A3, A3.5).

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT mix powder or liquid with any other glass ionomer components.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available for the Americas at: <http://www.gcamerica.com>
They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSER. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised: 09/2017

GC

MANUFACTURED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL : +1-708-597-0900
www.gcamerica.com

GC SOUTH AMERICA
Rua Heliodora, 399, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESP. TÉCN: Mayara de Santis Ribeiro - CRO/SP 105.982

GC ASIA DENTAL PTE. LTD.
11 Tampines Concourse, #03-05 Singapore 528729
TEL: +65 6548 7588

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