

ORTHO GEL CONDITIONER

CLEANING AGENT FOR ORTHODONTIC CEMENTS

Prior to use, carefully read the instructions for use.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

ORTHO GEL CONDITIONER is a polyacrylic acid gel solution (approx. 20%) designed to mildly etch and condition enamel, thus enhancing the bond strength of glass ionomer materials to tooth surfaces.

CONTRAINDICATIONS


In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

1. Hold the syringe upright and remove the wing cap by turning counterclockwise. Promptly and securely attach the brush tip by turning clockwise.
2. Clean tooth surfaces to be bonded with pumice and water. Rinse thoroughly with water
3. Dispense ORTHO GEL CONDITIONER directly onto the tooth surface to be bonded and leave for 10 seconds.
4. Rinse thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with an oil-free air. **DO NOT DESICCATE.** Prepared surfaces should appear moist (glistening).

NOTE:

- a. When attaching the brush tip, make sure the joint between the tip and the syringe is free of material.
- b. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

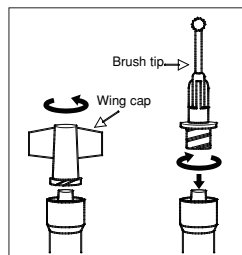

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).
(Shelf life: 2 years from date of manufacture)

PACKAGE

Syringe (1.4 g/1.2 mL)(3), Brush Tip (25)

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. The brush tip is disposable. The brush tip cannot be sterilized.



MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL: +65 6546 7588
PRINTED IN JAPAN