

SELF CONDITIONER

NO RINSE CONDITIONER FOR REINFORCED GLASS IONOMER

Prior to use, carefully read the instructions for use.

For use only by a dental professional for the recommended indications.

RECOMMENDED INDICATIONS

Conditioner for resin reinforced glass ionomer materials.

CONTRAINDICATIONS

In rare cases the product may cause some people sensitivity. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

Note: It is recommended to use rubber dam to isolate tooth.

1. Prepare the lesion using a standard technique. For pulp capping, use calcium hydroxide. Mechanical retention is not necessary. For a non cavitated Class V, clean the lesion with pumice and water, rinse thoroughly, and dry gently with an oil free air syringe.
 2. Dispense 1 or 2 drops of SELF CONDITIONER into a dispensing dish. Apply a thin layer of the conditioner to the bonding surfaces using a sponge or disposable micro-tip applicator, leave undisturbed for 10 seconds. Do not rinse away.
 3. Dry by gently blowing with an oil free air syringe for 5 seconds and ensure that the surface has a glossy appearance.
 4. Use a resin reinforced glass ionomer cement (GC Fuji Filling LC, GC Fuji II LC, GC FujiCEM, GC Fuji PLUS) for filling / luting procedure. Read the manufacturer's instructions for use carefully.
- Note:
- a. Use immediately after dispensing.
 - b. Should the applied conditioner be rinsed away, dry the cavity and apply the conditioner again.

PRINTED IN JAPAN
TEL: +82 0618 7288
19 Jyong Waj, #09-37 Singapore 508724
GC ASIA DENTAL PTE. LTD.
TEL: +1 708 587-0800
3737 West 127th Street, Aliso Viejo, CA 92686 U.S.A.
GC AMERICA INC.
TEL: +32 18 38.88.80
Research Park, Incheuvendries 13, B-3001 Leuven, Belgium
GC EUROPE N.V.
DISTRIBUTED BY
GC CORPORATION
78-1 Hasunuma-cho, Iizubashi-ku, Tokyo 174-8582, Japan
MANUFACTURED BY
GC CORPORATION
78-1 Hasunuma-cho, Iizubashi-ku, Tokyo 174-8582, Japan

STORAGE

Store at room temperature (4-25°C) (39.2-77.0°F), away from open flame, high temperature or direct sunlight.

Shelf life : 2 years from date of manufacture.

PACKAGE

SELF CONDITIONER

1 bottle 3 g (3.2 mL)

CAUTION

1. In case of contact with eyes, flush immediately with water and seek medical attention.
2. In case of contact with oral tissue or skin, immediately remove with a sponge or cotton pellet and rinse thoroughly with water.
3. If the tissue contacted by the conditioner turns white or forms a blister, advise the patient to leave the affected area undisturbed, until the mark disappears, usually in 1-2 weeks. To avoid gingival contact, it is recommended to apply cocoa butter to the area where rubber dam cannot cover.
4. Close the bottle immediately after use.
5. Avoid ingestion of the material.
6. If the conditioner is spilt onto a work surface or floor, immediately wipe off with a dry cloth.
7. Do not mix SELF CONDITIONER with any other products.

MANUFACTURED by

GC CORPORATION

76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by

GC CORPORATION

76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

GC EUROPE N.V.

Research-Park, Interleuvenlaan 13, B-3001 Leuven, Belgium

TEL: +32.16.39.80.50

GC AMERICA INC.

3737 West 127th Street, Alsip, IL 60803 U.S.A.

TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.

19 Loyang Way, #06-27 Singapore 508724

TEL: +65 6546 7588

PRINTED IN JAPAN

230161GC