Prior to use, carefully read the instructions for use.

## UNIVERSAL TRAY ADHESIVE

Adhesive for VPS Impression Materials and EXA'lence (Vinyl PolyEther Silicone [VPES] Impression Materials)

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

For adhering VPS impression materials or EXA'lence to impression trays.

### RECOMMENDED PRODUCTS

EXAMIX NDS, EXAFAST NDS, EXAJET, EXAFLEX and EXA'lence

\*All packaging types may not be available in all countries.

### CONTRAINDICATIONS

In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

1. Clean the inner surface of the tray and dry thoroughly.
2. Apply an even layer of adhesive onto the tray using a brush or cotton pellet.
3. Allow to stand for 5 minutes or air dry thoroughly with an air syringe.
4. Load the tray with impression materials.

### STORAGE

Store in a cool and dry place.  
(Shelf life: 3 years from date of manufacture.)

### PACKAGES

Universal Tray Adhesive 7mL bottle  
Universal Tray Adhesive 12-Pack (12-7mL bottles)

### CAUTION

1. In case of contact with skin or oral tissue, immediately remove with a sponge or cotton soaked in alcohol. Flush with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. The material is flammable and volatile. Do not use near naked flame. Keep away from sunlight or sources of ignition. Use in a well ventilated place.

Last revised: 07/2010

GC AMERICA INC.  
ALSIP, IL 60803 U.S.A.

MANUFACTURED by  
GC DENTAL PRODUCTS CORP.  
2-285 Toriimatsu-Cho, Kasugai, Aichi 486-0844,  
Japan

DISTRIBUTED BY  
GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803, U.S.A.

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