

initial Vita[®] 3D Master Mixing Chart for GC Initial[™] MC








Color 3D	Opaque	Dentin	Ratio	Enamel
	A1	D2 + BLD-1	1:1	E-58
	A1	A1 + IN-44	6:1	E-58
	A2	A1 + IN-51	6:1/2	E-58
	B2	B2 + IN-47	5:1	E-58

[†]Not a registered trademark of GC America Inc.

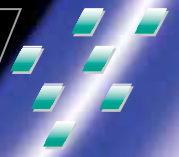

initial Vita®† 3D Master Mixing Chart for GC Initial™ MC



Color 3D	Opaque	Dentin	Ratio	Enamel
	A1	C1 + BLD-1 + IN-47	4:1:1/2	E-58
	A1	A1 + IN-45	7:1	E-58
	B2	B2 + B3	1:1	E-59
	A1	A1 + IN-41	5:1	E-59

†Not a registered trademark of GC America Inc.



initial Vita[®]† 3D Master Mixing Chart for GC Initial™ MC



Color 3D	Opaque	Dentin	Ratio	Enamel
	A2	A2 + IN-47	5:1/2	E-59
	D3	C1 + IN-46	6:1	E-59
	B3	B3 + A3	5:1	E-60
	D3	D2 + C4	5:1	E-59

†Not a registered trademark of GC America Inc.



initial Vita^{®†} 3D Master Mixing Chart for GC Initial[™] MC







Color 3D	Opaque	Dentin	Ratio	Enamel
	D3	D3 + IN-45	6:1	E-60
	B4	B4 + IN-42	6:1	E-60
	A4	D2 + IN-41	6:3	E-60
	A3,5	A3 + IN-42	4:2	E-60

[†]Not a registered trademark of GC America Inc.



initial Vita^{®†} 3D Master Mixing Chart for GC Initial[™] MC



Color 3D	Opaque	Dentin	Ratio	Enamel
	D3	D3 + C4	3:1	E-60
	A4	A4 + IN-50	5:1	E-60
	A2	D3 + CT-25 + TM-05	2:1:3	E-59
	A2	A3 + IN-42 + TM-05	3:4:5	E-59

[†]Not a registered trademark of GC America Inc.



initial Vita[®] 3D Master Mixing Chart for GC Initial[™] MC




Color 3D	Opaque	Dentin	Ratio	Enamel
	A4	A3,5 + IN-45 + IN-48	5:1:1/2	E-60
	A2	D3 + CT-25 + TM-05	3:2:2	E-60
	D3	A3,5 + IN-45 + TM-05	3:2:2	E-60
	A4	D3 + CT-23 + TM-05	3:1:3	E-60

initial Vita[®] 3D Master Mixing Chart for GC Initial[™] MC



Color 3D	Opaque	Dentin	Ratio	Enamel
	A4	A4 + CT-23 + TM-05	2:3:2	E-60
	A4	A4 + IN-49 + IN-48	5:1/2:1/2	E-60

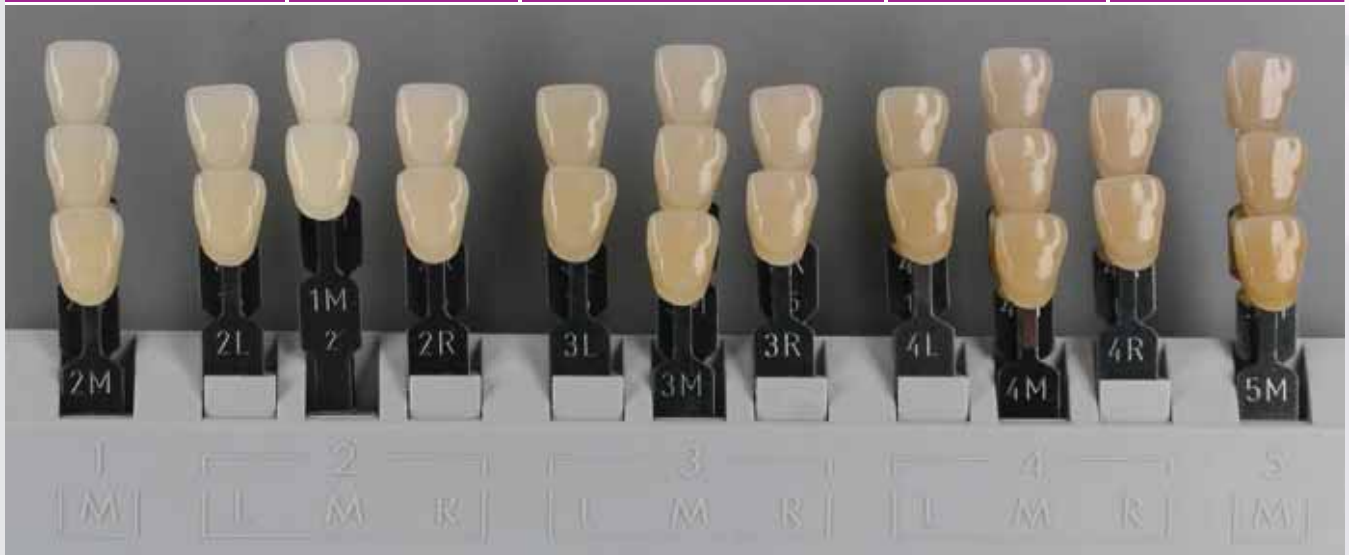

[†]Not a registered trademark of GC America Inc.



initial Vita^{®†} 3D Master Mixing Chart for GC Initial[™] MC



Vita ^{®†} 3D Master	Vita ^{®†} Classic (GC Initial MC)	Ratio	Opaker	Enamel
1M1	D-D2 + BLD-1	1:1	A1	E-58
1M2	D-A1 + IN-44	6:1	A1	E-58
2L1,5	D-A1 + IN-51	6:1/2	A2	E-58
2L2,5	D-B2 + IN-47	5:1	B2	E-58
2M1	D-C1 + BLD-1 + IN-41	4:1:1/2	A1	E-58
2M2	D-A1 + IN-45	7:1	A1	E-58
2M3	D-B2 + D-B3	1:1	B2	E-59
2R1,5	D-A1 + IN-41	5:1	A1	E-59
2R2,5	D-A2 + IN-47	5:1/2	A2	E-59
3L1,5	D-C1 + IN-46	6:1	D3	E-59
3L2,5	D-B3 + D-A3	5:1	B3	E-60
3M1	D-D2 + D-C4	5:1	D3	E-59
3M2	D-D3 + IN-45	6:1	D3	E-60
3M3	D-B4 + IN-42	6:1	B4	E-60
3R1,5	D-D2 + IN-41	6:3	A4	E-60
3R2,5	D-A3 + IN-42	4:2	A3,5	E-60
4L1,5	D-D3 + D-C4	3:1	D3	E-60
4L2,5	D-A4 + IN-50	5:1	A4	E-60
4M1	D-D3 + CT-25 + TM-05	2:1:3	A2	E-59
4M2	D-A3 + IN-45 + TM-05	3:4:5	A2	E-59
4M3	D-A3,5 + IN-45 + IN-48	5:1:1/2	A4	E-60
4R1,5	D-D3 + CT-25 + TM-05	3:2:2	A2	E-60
4R2,5	D-A3,5 + IN-45 + TM-05	3:2:2	B4	E-60
5M1	D-A4 + CT-23 + TM-05	3:1:3	D3	E-60
5M2	D-A4 + CT-23 + TM-05	2:3:2	A4	E-60
5M3	D-A4 + IN-49 + IN-48	5:1/2:1/2	A4	E-60