initial

FIRING FOAM

GC Initial FIRING FOAM

EN Instructions for use

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Master all your challenges

FEBRUARY 2016

GC

GC Initial FIRING FOAM

Special refractory support material for ceramic firing

The GC Initial FIRING FOAM for all GC Initial ceramic systems ensures a stable fixing of objects onto the ceramic firing trays.

INDICATIONS

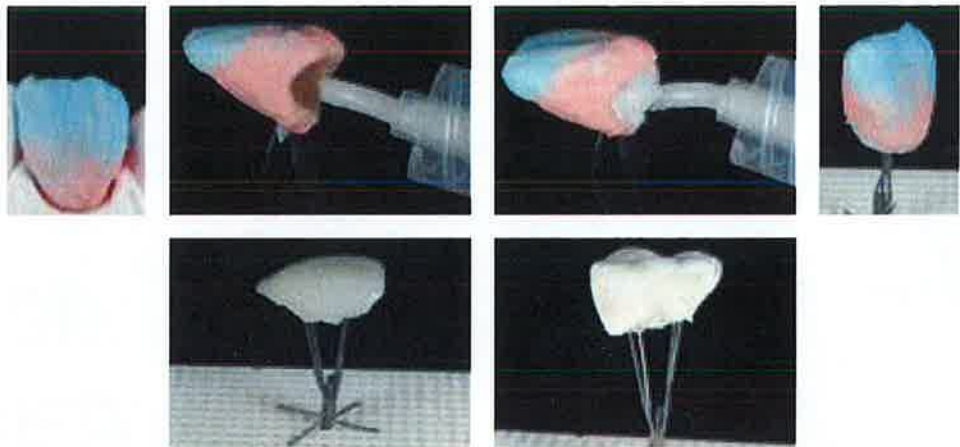
- Inlays, onlays, crowns, veneers, bridges

BENEFITS

- Ease of application due to its special creamy consistency
- Guaranteed dimensional stability during the firing process
- Perfect distribution of heat onto the object
- Easy to remove after firing

DIRECTIONS FOR USE

APPLICATION

- Apply FIRING FOAM directly from the syringe into the crown or onto the pressed ceramic object
- Carefully place crown or pressed ceramic object on the pin of the ceramic firing tray

REMOVAL

- After firing, quickly remove the solidified FIRING FOAM under running water with an instrument
- If necessary, blast off with non-abrasive glass beads under a maximum pressure of 2 bar

REMARK

Avoid contact of the FIRING FOAM to the ceramic. Use only as much FIRING FOAM as is needed. When using low fusing ceramic types (e.g. GC Initial LF / GC Initial LiSi) increase the drying time up to 8 minutes.

STORAGE

Well close the syringe to ensure an airtight seal. Store in a cool and dry place.

PACKAGES

1x GC Initial FIRING FOAM, 10g

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas: <http://www.gcamerica.com>. They can also be obtained from your supplier.

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- Inlays, Onlays, Kronen, Veneere, Brücken

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GC America Inc., Alsip, IL 60803 U.S.A.