Prior to use, carefully read the instructions

EN

GC Initial INmetalbond

Buffer between GC Initial metal ceramics and dental alloys.

GC Initial INmetalbond

INmetalbond is used as a thin layer between the alloy and the first Opaque layer.

The INmetalbond blocs the escaping metal oxides and neutralizes dif-ferences in the expansion coefficient.

The bonder does not increase the bonding strength as such but allows a wider span of CTE compatibility.

Recommended indications

Can be used on all PFM alloys, precious as well as non-precious.

Instruction For Use

- Metal framework needs to be prepared according manufacturer’s instructions.
- Oxidation firing again according manufacturer’s instructions.
- Stir paste before using.
- Apply the Bonder in a thin layer but mask the metal completely. **IMPORTANT:**
 - To change the consistency of the INmetalbond paste, use the “GC Initial Paste Opaque Thinner”.
 - Only use it in very small quantities.
 - To avoid drying out, close the cap after using.
- The fired Bonder should have a yellowish, slightly shiny surface. **NOTE:**
 - The colour of the fired bonder can vary depending on the composition of the alloys.
 - When using NPA (non-precious alloys) with the INmetalbond, the final temperature of the Opaque Washbake does not need to be increased by 20°C anymore. e.g.: 1st Opaque Firing on NPA: GC Initial MC = 940°C GC Initial LF = 830°C

Firing instructions

Preheating Temp.	Drying Time	T° increase	Vacuum	Final Temp.	Holding Time
550°C	6min	80°C/min	Yes	980°C	1min no vacuum

Storage

Store in a cool, dry place.

Shelf life

GC Initial INmetalbond: 5 years

Packages

GC Initial INmetalbond, 2x4gr.

Last revised: 05/2013

CE 0086

GC

MANUFACTURED by
GC EUROPE N.V.
Research-Park Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

 : GC EUROPE N.V.
Research-Park Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
11 Tampines Concourse, #03-05, Singapore 528729
TEL: +65 6546 7588

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