Initial^{Zr}

APRIL 2011

LIGHT REFLECTIVE LINERS

Technical Manual / Verarbeitungsanleitung
Manuel Technique / Manuale Tecnico
Manual Técnico / Instrukcja techniczna

GC Initial Zr Light Reflective Liners

GC

*initial*_{Zr}

Light Reflective Liners

GC Initial Zr Light Reflective Liner Set

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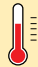
Note: Ceramic furnaces of different types may vary significantly in real firing temperature compared to indicated temperature. Please periodically calibrate your furnace and do test firings if necessary.

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Important Information:

All GC Initial, IQ One Body Layering-over-Zircon materials and GC Initial Zr-FS materials are fully compatible and can easily be combined for extended options.

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GC Initial • Zircon - Light Reflective (LR) Liners

GC Initial • Zircon - Light Reflective (LR) Liners

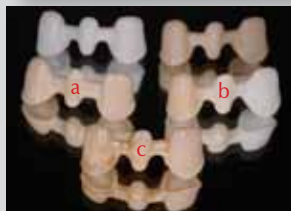
Using the Zr-LR Liners, you can easily adjust the shade of different coloured ZrO₂ materials for a systematic approach. For example, to achieve the Classic Vita shades, see the colour combination charts page 5.

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A single-colour application (a) of the Zr-LR Liners is possible, as well as a multi-colour application (b), or an additional coloration with GC Initial IQ Lustre Pastes NF (c).

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GC Initial • Zircon - Light Reflective (LR) Liners

GC Initial • Zircon - Light Reflective (LR) Liners

Apply Zr-LR Liner evenly thin, but covering.
Refer to the firing chart on page 5.

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Tip: Use more intense Zr-LR Liner to modify the basic shades partially. To achieve maximum modification, we also recommend the use of GC Initial IQ Lustre Pastes on the Liner.

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initial^{Zr}

Manuel Technique
Manuale tecnico

GC Initial Zr Light Reflective Liners

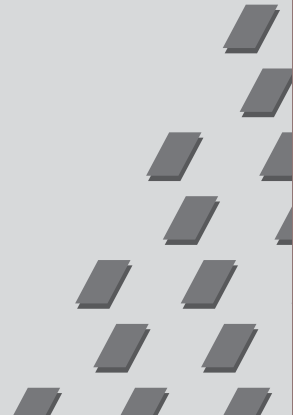
Table des matières

Indice

	Page / Pagina
GC Initial • Zircon - Light Reflective (LR) Liners, tableaux de combinaison des teintes & tableau de cuisson /	
GC Initial • Zircon - Light Reflective (LR) Liners, Tavole delle combinazioni cromatiche & Tabella delle cotture	11
GC Initial • Zircon - Light Reflective (LR) Liners, Propriétés physiques et péremption /	
GC Initial • Zircon - Light Reflective (LR) Liners, Proprietà fisiche e durata utile	12
GC Initial • Zircon - Light Reflective (LR) Liners , application /	
GC Initial • Zircon - Applicazione del Liner Riflettente	13-14

Photos / Fotografie:

ZTM Michael Brüsch - M.B. Dentaltechnik GmbH, Düsseldorf, Germany

ZTM Oliver Heinze, Götzis, Austria

CDT Mark Bladen, Worcestershire, United Kingdom

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Vita® is a registered trademark of Vita Zahnfabrik H. Rauter GmbH & Co. KG, Bad Säckingen, Germany.
GC NORDIC AB
GC Nordic AB
GC AUSTRIA GmbH