initial IQ

GC Initial IQ
Lustre Pastes NF
Gum Shades

EN Technical Manual

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FR Manuel Technique

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ES Manual técnico

PT Manual técnico

Master all your challenges

OCTOBER 2017

GC

- EN True-to-nature ...
- DE Naturidentisch ...
- FR Proche du naturel ...
- IT Estetica naturale ...

Photoshooting 1 - 4: CDT D. Ibraimi, Switzerland
 Photoshooting 5 - 6: Byrnes Dental Laboratory, UK

EN	DE	FR	IT	
Table of Contents	Inhaltsverzeichnis	Table des matières	Indice	Page
Intended Use and Shelf Life	Bestimmungsgemäßer Gebrauch und Haltbarkeit	Utilisation et péremption	Campo di impiego e durata dei prodotti	4-5
Description of the Lustre Pastes NF Gum Shades	Beschreibung der Lustre Pastes NF Gum Shades	Description des Lustre Pastes NF Gum Shades	Descrizione delle Lustre Pastes NF Gum Shades	
Basic colours	Basis-Farben	Teintes de base	Colori di base	6-7
Effect colours	Effekt-Farben	Effets de teinte	Colori per effetti	
The liquids of the Lustre Pastes NF Gum Shades Set	Die Flüssigkeiten des Lustre Pastes NF Gum Shades Set	Les liquides du coffret Lustre Paste NF Gum Shades	I liquidi del set di Lustre Paste NF Gum Shades	8-9
Lustre Pastes NF Gum Shades Application	Lustre Pastes NF Gum Shades Anwendung	Application Lustre Pastes NF Gum Shades	Applicazione delle Lustre Pastes NF Gum Shades	
Basic Step by Step Instruction (Zirconium Frame)	Basis-Farben Step by Step Anleitung (Zirkon-Gerüst)	Etape par Etape (armature zircon)	Istruzioni fondamentali passo per passo (frame in zirconia)	10-15
Basic Instruction (Non Precious & Ti Dental Frame)	Basis-Farben Anleitung (Nichtedelmetall- / Ti-Gerüst)	Etape par Etape (armature dentaire non précieuse / armature Titane)	Istruzioni di base (frame in leg vile / struttura in Titanio)	16-17
Firing Chart	Brenntabelle	Tableau des températures	Grafico delle cotture	18
Tips and Tricks	Tipps und Tricks	Trucs et astuces	Consigli e accorgimenti	19
Facilities of the Lustre Pastes NF Gum Shades	Möglichkeiten mit Lustre Pastes NF Gum Shades	Exemples d'application des les Lustre Pastes NF Gum Shades	Esempi di applicazioni delle Lustre Pastes NF Gum Shades	20-21
FAQ	FAQ	FAQ	Domande più frequenti	22-23

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EN Paint-on your gum shades

GC Initial IQ Lustre Pastes NF Gum Shades are 3-dimensional ceramic stains developed to enable dental technicians to create a lifelike transition from the crown margin to the gingiva just by painting the gum shades. With a continuously growing demand for aesthetic implant-supported superstructures, there has also been a growing desire among dental technicians for a ceramic staining approach for the soft tissue zones. With the new unique Lustre Pastes NF Gum Shades Set, you can from now on bring not only the white, but also the red aesthetics of your crowns and bridges to life. These unique gum coloured Lustre Pastes allow an individual adaptation to match natural gingival shades offering a natural harmony. With their optimal match to both "low" and "high" CTE ceramics, the system is compatible with the complete GC Initial ceramic range. Available in a ready to use consistency, the Lustre Pastes NF Gum Shades are based on fine ceramic particles and can be applied in a thicker layer where they will exhibit unsurpassed nature analogue gum reproduction and a very natural glaze to your restorations.

Features

- One kit for the complete GC Initial ceramic range.
- No complex layering or extra firing cycles required. Fast – Easy – Secure.
- Available in ready to use jars for a more convenient application.
- Diluting and Refresh liquids to keep the optimal consistency until the end.
- Five gum coloured pastes with an ideal thixotropy for easy and precise application.
- Four mixable Gum Modifiers allowing unlimited creativity to copy natural gingival harmony.

Indications

The Lustre Pastes NF Gum Shades are a one for all solution for the complete Initial line from GC. It's possible to use the Lustre Pastes NF Gum Shades in a wide CTE – Range in order to work with nearly all types of ceramic. These Lustre Pastes NF Gum Shades represent a modern alternative for conventional staining of gingival areas. Due to the unique properties it's just a question of minutes to individualize the gingival areas of your crown and bridge work.

Keep in mind: What you paint (on) is what you get (after firing)

GC Initial IQ Lustre Pastes NF Gum Shades Set

Contents:

1x Base Light, 1x Base Dark, 1x Intensive Violet, 1x Intensive Red, 1x Intensive Cream, 4x Modifier (white, red, blue, brown), 1x Diluting Liquid, 1x Refresh Liquid, 1 mixing dish, 2 brushes, 1x Technical Manual, 1x Colour Chart

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EN Note: Mix the material well before use.

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LP-M1
Modifier White

EN Indications:

The modifiers can be used to intensify the single Lustre Pastes Gum Shades allowing an unlimited reproduction of gum colours.

Instructions for use:

The different modifiers can be added & mixed directly with the Lustre Pastes Gum Shades. The desired colour can visually be controled.

When adding modifiers to the Lustre Pastes Gum Shades there is no need to adapt the firing parameters.

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● EN Lustre Paste 'Diluting Liquid':

The 'Diluting Liquid' is a special liquid to change the flowability of the Lustre Pastes. However, it is recommended to obtain a gel/paste like consistency of the Lustre Pastes for best results.

Attention:

- If the Lustre Pastes are too much diluted, the result is a reduction in the gloss of the paste after the firing; hence, the desired result may not be achieved in one glaze firing.
- When needed, dilute the desired amounts of the pastes in a separate plate with Diluting Liquid, never dilute directly in the jars.

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● EN Lustre Paste 'Refreshing Liquid':

The 'Refreshing Liquid' is used to preserve the thixotropic property of the pastes. This liquid is used to recover dried out pastes to an optimum, gel like consistency without changing the physical properties. The Refresh Liquid can be used both in the jars and mixed on a separate plate.

Important: Please shake the Refresh Liquid well before use!

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● **EN** In the following step by step instruction the use of the Lustre Pastes NF Gum Shades are demonstrated on a zirconium framework. Please note that some of the used ceramic products are not included in the Lustre Pastes NF Gum Shades Set. Therefore we refer you to their dedicated Technical Manuals /IFU: GC Initial Zr-FS/ Zr Light Reflective Liners/IQ Layering-Over-Zircon/IQ Lustre Pastes NF system.

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● **EN** Framework preparation
 • Prepare the framework for ceramic veneering following the manufacturer's Instruction For Use.
 • For GC Initial users, refer to the GC Initial Zr-FS Technical Manual.

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● **EN** The Light Reflective Liner is applied in a thin covering layer to the tooth-coloured area.

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EN After the second firing the restoration can be shaped & contoured in the usual way. Before applying the Lustre Coating, the ceramic restoration is shaped and contoured using adjusted diamond burs. After polishing with a ceramic polisher the finished Crowns & Bridges are sand-blasted with 50 microns Alu-Oxid at 1,5 bar pressure.

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EN To intensify the areas between the tooth roots, e.g. Lustre Paste G-36 can be used.

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● **EN** For more detailed instructions, please refer to the dedicated Technical Manuals / IFU of each of the below GC Initial Lines:

- GC Initial MC/LF
- GC Initial Ti
- GC Initial Ti Bonder
- GC Initial IQ Layering-over-Metal
- GC Initial IQ Press-over-Metal
- GC Initial INmetalbond

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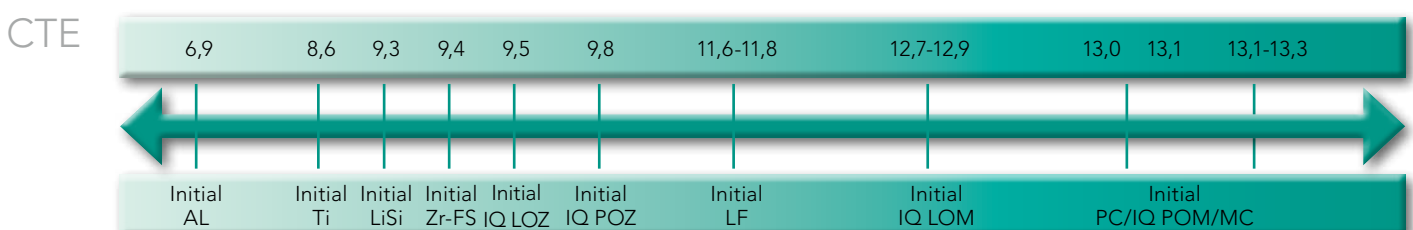
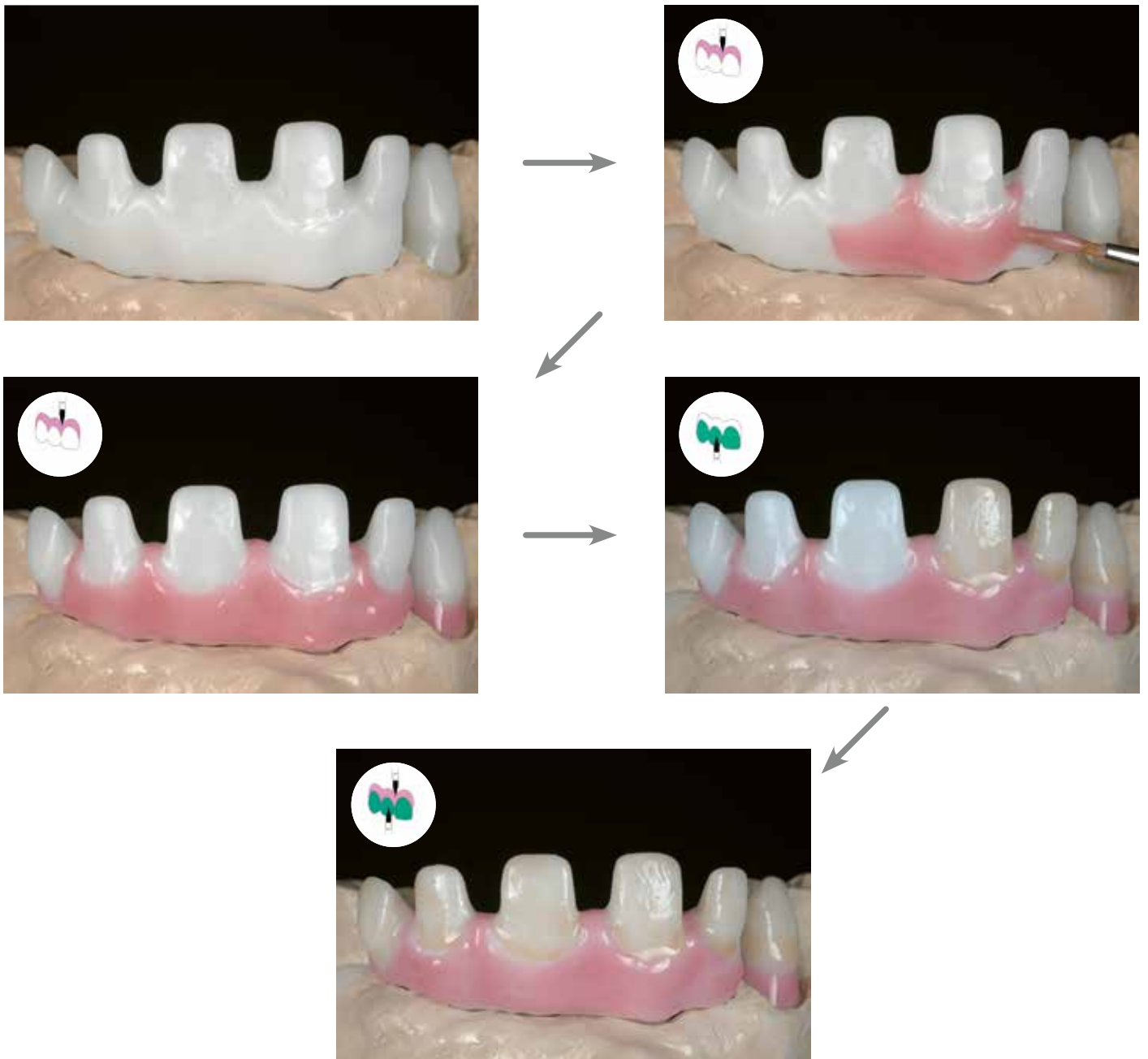
DE FAQ

FR FAQ

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EN Basic Rule: What you paint (on) is what you get (after firing)

Q1. How can I dilute (adapt the consistency of) the Lustre Pastes NF Gum Shades?

Use the Lustre Paste NF Diluting Liquid to adapt the consistency.

Do NOT use water to dilute. LP may not come in contact with water (see page 8).

Q2. What can I do when the consistency of the Lustre Pastes NF Gum Shades is too dry?

Use the Refresh Liquid to recover the Lustre Pastes in case of dryness (see page 9).

Q3. How do we change the texture of the Lustre Pastes NF Gum Shades?

The requested surface texture/smoothness of the Lustre Pastes NF Gum Shades layer can be changed by soft vibration or condensing the restoration.

Q4. I do not have a shiny appearance after firing the Lustre Pastes NF Gum Shades, what is the cause?

Check the firing temperatures, increase the final temperature. When the Lustre Pastes NF Gum Shades are heavily diluted or applied in a too thin layer the glossy effect is tempered, so apply a more thick layer/dilute less the pastes.

Q5. Can I apply more than one layer of Lustre Pastes NF Gum Shades?

Yes, you can do a second firing, keeping the same firing temperatures.

Q6. Can I mix the Lustre Pastes NF Gum Shades with other Initial powders?

You can use the GC Initial Modifier powders to intensify the Lustre Pastes NF Gum Shades.

Mix the required amount of Modifier powder with the LP Dilution Liquid and mix it with the required LP or directly mix the Modifier powders with e.g. the Lustre Pastes NF Gum Shades.

Q7. Are the Lustre Pastes NF sufficient abrasion resistant?

Yes, the LP NF Gum Shades are a special mixture of different types of low fusing ceramic particles, stains & glaze mixture offering lifelike aesthetics.

Q8. May I use the Lustre Pastes NF Gum Shades for AL in the same way I use it for other materials?

No, you should apply and fire the Lustre Pastes NF Gum Shades in 2 thin layers.

Q9. The glaze firing is blearily, spotted or cracked?

The Lustre Pastes NF Gum Shades came partly or completely in contact with water.

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GC NORDIC AB
GC AUSTRIA GmbH