Prior to use, carefully read the instructions for use.

(EN)

METALPRIMER II

PRIMER FOR BONDING RESIN TO METAL

METALPRIMER II contains a special monomer (MEPS : thiophosphoric methacrylate) which promotes bonding of acrylic resins and composites to all types of metal.

RECOMMENDED USE

Bonding of acrylic and composite resins to all types of metal as set out below in the DIRECTIONS FOR USE.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE

GENERAL

In all cases

- Sandblast metal surfaces with alumina (aluminium oxide), and blow clean with air gun.

Note :

- Avoid contamination with oil, separator or moisture from fingers, etc.
- Dispense 2-3 drops of METALPRIMER II into a dappen dish or similar container. Using the brush provided, apply a thin layer to the bonding surface. Allow to dry.

Note :

A second coat may be applied if necessary.

- Apply resin to metal as indicated below.

Note :

Apply resin to metal immediately after the surface is prepared.

RECOMMENDED APPLICATIONS

- Resin cement to metal for metal inlays, crowns and bridges
Prepare metal surface as above and bond restoration with resin cement.
- Opaque resin to metal for composite resin veneers and crowns
Prepare surface as above and apply opaque (GC GRADIA) in usual manner. Cure as recommended by the manufacturer.
- Denture base to magnetic attachment
Prepare surface as above and bond attachment in usual manner using self curing acrylic resin (GC UNIFAST Trad).
- Denture base resin to metal base, clasp or attachment
(a) Prepare bonding surface with alumina sandblast and blow clean with air gun.
(b) Form a wax denture.
(c) After flushing out the wax, clean the bonding surface using a steam cleaner. Remove wax completely using solvent.
(d) Apply METALPRIMER II
- (e) Pack and cure denture base in usual manner.
- In the mouth repair of damaged crown and bridge veneers
(a) Place rubber dam.
(b) Clean metal surface with carbide bur and prepare as above.
Special note :
Be sure to use rubber dam and avoid direct contact with oral tissue. In case of contact, immediately flush with water and seek medical attention.
- Repair of denture with metal base, clasp or attachment
(a) Roughen metal bonding surface using carborundum point.
(b) Prepare bonding surface as above.
(c) Build up using self-cure resin in the usual manner.
- Bonding resin to metal orthodontic appliances
(a) Prepare bonding surfaces as above.
(b) Apply orthodontic resin to the appliance in the usual manner.

PACKAGE

1 bottle (5 mL) METALPRIMER II
1 brush (No. 7)

STORAGE

Recommended for optimal performance, store in a cool place (4-25°C)(39.2-77.0°F) out of direct sunlight.

Shelf-life: 2 years from date of manufacture

CAUTION

- For use only by dental professionals for the applications recommended.
- Contains methacrylate. Not to be used by operators, or on patients, with a history of allergy to methacrylates. In cases of allergic response, seek medical attention.
- In case of contact with skin or eyes, immediately flush with water and seek medical attention.
- Flammable. Avoid high temperatures, direct sunlight and sources of ignition. Use in a well ventilated area.
- Volatile. Seal container immediately after use.
- Assign the brush provided (No. 7) for use ONLY with METALPRIMER II
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

http://www.gceurope.com

or for The Americas

http://www.gcamerica.com

They can also be obtained from your supplier.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised : 09/2017

CE 0086

'GC'

MANUFACTURED by
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan
RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED BY
GC CORPORATION
761-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

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