Prior to use, carefully read the instructions for use.

EN

NEW FUJIROCK[®] FAST SET

IMPROVED DENTAL STONE

NEW FUJIROCK FAST SET is a dental stone designed to set in only about 5 minutes.

FEATURES

1. Sets in only about 5 minutes, yet provides proper compressive strength in a shorter time than the conventional ones. Enables work to be done more quickly.
2. Provides ample working time and sharp set.
3. In pouring, runs less from impressions. Exhibits superior handling characteristics.
4. Minimum setting expansion provides for accurate models with smooth surfaces.

APPLICATIONS

1. Models for inlays, 3/4 crowns, jacket crowns, post crowns, bridges.
2. Models for full or partial dentures & for repairs.
3. Models for temporary restorations.
4. Orthodontic models.

DIRECTIONS FOR USE

1. Standard water / powder ratio: 20mL/100g (one full measure of water to 2 spoonfuls of powder).
2. Mix uniformly for 30-60 seconds.
3. Pouring cast.

The Vibrator is highly recommended to get a model without any porosity.

4. Removal of model from an impression.
Alginate impression material: about 12 minutes after pour.
Polysulfide rubber or Silicone rubber impression material: 15 minutes after pour.

NOTES

1. Wear a dust mask to avoid inhaling powder dust.
2. Be careful not to get powder into eyes. In case of contact, immediately flush with water, and consult a physician.
3. When cutting and trimming hardened stone, wear goggles for eye protection.
4. After opening, transfer contents into a container with a lid. After use, tightly close the container. Store in a cool, dry place.
5. Do not mix the powder with any other dental stone
6. This product should be used only for the applications described in the instructions.
7. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

PHYSICAL PROPERTIES	
ITEMS	VALUE
Water/powder ratio (W / P)	0.20
Setting time (min)	5
Linear setting expansion after 2 hr	0.08
Compressive strength after 1 hr	58.0
JIS T 6605	

Last revised: 5/2013

GC

MANUFACTURED by
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

PRINTED IN U.S.A.

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

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