

WAX-RITE™ INSTRUCTIONS

To start unit, turn Thermostat knob mid-way between "OFF" and "MAX." position. Place wax in reservoir. When unit is heating, Thermostat Pilot Light will glow red. When wax reaches temperature selected with Thermostat, Pilot Light will go out. Once unit reaches desired temperature, Pilot Light will glow only when additional current is needed to maintain wax at proper temperature. If more heat is required, turn Thermostat clockwise; if less heat is required, turn Thermostat counter-clockwise.

For maximum safety and economy, unplug unit at the end of the day.

note

1. Do not immerse unit in water.
2. When unplugging unit, do not pull cord; grasp plug.
3. To keep wax clean, close cover when not in use.
4. No operator serviceable parts inside unit.

Do not open. Contact GC Lab Technologies for servicing information.



GC Lab Technologies

3737 WEST 127TH STREET
ALSIP, ILLINOIS 60803