ALIKE

Prior to use, carefully read the instructions for use.

EN

ALIKE is a tooth-colored, self-curing resin for the construction of temporary bridges and crowns.

WARNING: "An in mouth period of 2:00 (120 s) is sufficient to provide impetus for quick completion of reaction. The reaction should be complete within 2:00 of time of removal from mouth, more quickly if impression is placed in warm water."

NOTE:

It is normal for the color of the Alike liquid to change to a darker color as it ages. This change in liquid color has an insignificant effect on cured acrylic color.

Direct Method

- Before preparing the teeth, take a full mouth impression. EXAMIX/EXAFLEX or CHG Alginate may be used for the impression. The CHG Alginate Impression should be wrapped in a wet towel until ready for use.
- Complete all necessary mouth preparations.
- Cut away sufficient alginate or silicone in all pontic areas of the impression to provide continuity between the abutment teeth. This space will later be filled with ALIKE to serve as temporary replacement for the missing teeth. Anatomy and contour can be perfected after the ALIKE has cured.
- If the impression was made with CHG Alginate, rinse the impression with warm water. Remove excess water from the impression with gentle air spray. Select the shade of ALIKE Powder desired. Place a small amount of ALIKE Liquid in all abutment teeth and pontic areas. Add sufficient ALIKE Powder to absorb the liquid. Abutment and pontic areas should be slightly over-filled. Powder and liquid may also be mixed in a mixing cup. Use the proportions: 1 part liquid to 3 parts powder by volume. Stir the mixture rapidly with a cement spatula for 10 to 15 seconds. Quickly pour the mixture into the abutment teeth and pontic areas of the impression. Average working time for pouring is brief, approximately 30 to 40 seconds.
- Wait approximately 30-60 seconds until the ALIKE mixture reaches the desired flow. Coat the prepared teeth and adjacent gingival tissues with a thin film of vaseline, which serves as a lubricant for easy removal of the ALIKE temporary bridge.
- Place the impression in the mouth with pressure. Remove the impression from the mouth before the exothermic heat begins, approximately 2 minutes. Allow the ALIKE temporary bridge to bench cure in the impression. Do not allow ALIKE to polymerize on the abutment teeth. To accelerate curing, immerse the impression in warm water approximately 110°F.
- Remove the ALIKE bridge from the impression. Trim and polish as necessary. Carefully fit the bridge over the abutment teeth and establish proper occlusion.
- Cement the bridge with FREEGENOL OR NOGENOL Temporary Cement. Proper occlusion should also be established after cementation. ALIKE may also be used with all plastic crown forms.

Indirect Method

- The ALIKE temporary bridge is constructed on a diagnostic cast or study model. COECAL dental stone should be used for minimum dimensional change in setting, extreme hardness, and for fidelity in reproducing fine detail. Edentulous spaces are filled with waxed teeth or stock plastic teeth that have been ground to make passive contact with the oral mucosa. Occlusion of the pontics can be established by articulating the cast with a model of the opposing arch.
- Soak the cast in room temperature water for 5 minutes. Take an impression with EXAMIX/EXAFLEX of CHG Alginate. The CHG Alginate impression should be wrapped in a wet towel until ready for use.
- Remove all pontics from the stone model. Cut a thin layer of stone approximately 0.5 mm from all the abutment teeth, so each tooth resembles the final crown preparation, but with a greater diameter. Resoak the cast in warm water for 5 minutes. Paint a thin layer of COE-SEP tinfoil substitute on the prepared teeth and surrounding tissues.
- Select the desired shade of ALIKE powder. In a mixing cup, add 1 part liquid to 3 parts powder. Stir the mixture quickly with a cement spatula for 10 to 15 seconds. Immediately, pour the mixture into abutment teeth and pontic areas in the impression.
- Wait approximately 30-60 seconds until the ALIKE mixture reaches the desired flow. Place the stone cast into the impression with pressure. Remove the cast from the impression in approximately 3 minutes. Carefully separate the ALIKE temporary bridge from the stone cast (or the impression) and allow it to polymerize on the bench. (3 minutes)
- Trim and polish the ALIKE bridge as desired. Place the finished bridge in water until needed.
- After the tooth preparations are completed, try the ALIKE temporary bridge over the prepared abutments. Trim the bridge as necessary to obtain proper fit.
- Lubricate the prepared teeth and surrounding gingival tissues with a thin film of vaseline. Prepare another ALIKE mixture (1 part liquid to 3 parts powder). Using a cement spatula, place the mixture into the abutment areas of the bridge. Place the bridge over the abutments with pressure. Remove the bridge from the mouth before the exothermic heat begins. (Approximately 2 minutes). Allow the ALIKE temporary bridge to bench cure. (3 minutes)
- Finish and polish the temporary bridge as desired.
- When fit and occlusion are satisfactory, cement the ALIKE bridge with FREEGENOL or NOGENOL Temporary Cement. The occlusion should be examined after cementation.

Irritation and sensitization have been reported in conjunction with products that contain methacrylates. To reduce the risk of such occurrences, do not use this product with patients who have demonstrated sensitivity, avoid repeated and prolonged contact with uncured material and discontinue use if sensitization occurs.

STORAGE CONDITIONS

Store in a dry location at room temperature (70° to 77°F; 21° to 25°C).

PACKAGES

Alike Professional Package
Alike Powder Refill Bottle 1 1/2 oz. (45g)
Alike Powder Refill Bottle 6 oz. (170g)
Alike Liquid Refill Bottle 4 oz. (118mL)

SHELF-LIFE

Liquid and Professional Package = 24 months
Powder Only = 36 months

Handling Precautions

Flammable liquid. Keep away from heat, sparks and flame. Use with adequate ventilation. Do not take internally or breathe vapor. Protective eye wear is required before using this product. In case of contact, flush eyes with water for at least 15 minutes. Get prompt medical attention. If overcome by inhalation, remove to fresh air. If not breathing, apply artificial respiration. If breathing is difficult, administer oxygen. Call a physician. If swallowed, dilute by giving water to drink and call a physician. Never give anything by mouth to an unconscious person.

Last revised: 01/2012

MANUFACTURED BY
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

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