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ZOE IMPRESSION PASTE

GB**TAKING THE IMPRESSION**

1. Instruct the patient to rinse the mouth thoroughly to remove excess saliva.
2. Lightly lubricate the patient's lips with petroleum jelly.
3. The standard mixing ratio of base to accelerator is 1:1 (by weight). This can be achieved by extruding equal lengths of paste from each tube onto a Coe mixing pad. Recap each tube.
4. Normal working temperature is 22°C (72°F) to 25°C (77°F). Using a Coe stainless steel spatula, mix the two pastes with broad sweeps. A complete mix of uniform color can be easily obtained in 30-45 seconds.
5. Spread the mixture over dry surfaces of the prepared tray.
6. Place tray in the mouth. Begin muscle trimming in the usual manner approximately 30 seconds after seating.
7. COE-FLO sets hard in the mouth in 3-4 minutes. After insertion, verify complete set and remove the impression by instructing patient to blow out cheeks.

FOR FASTER SET—Stir 2 to 3 drops of ethyl alcohol into the brown paste before mixing. This will not affect the accuracy of the impression.

FOR SLOWER SET—Add 1 to 2 drops of glycerine to the mix. Mix thoroughly.

CORRECTING THE IMPRESSION

1. Thoroughly dry the impression.
2. Add a sufficient amount of softened wax to the impression to fill the void. Do not overfill the defect.
3. While the wax is soft, replace the impression in the mouth.
4. Hold the impression gently in place for 3 to 4 minutes to allow the wax to harden.
5. Remove the impression and prepare the model.

POURING THE COECAL CAST

1. Rinse Coe-Flo impression thoroughly under cold running water. Shake excess water from the impression.
2. Mix Coecal and gently vibrate into impression. No separating medium is required.
3. To separate cast from impression, immerse in hot water (120° to 140°F) for 5 to 10 minutes. Do not allow water to boil.

FOR STABILIZING BASE PLATES

1. Fill extreme undercuts on cast with wax and immerse in warm water for 10 minutes.
2. Mix Coe-Flo and spread evenly over dry tissue surface of base plate.
3. Place the cast carefully into the formed base plate, anterior portion first.
4. Trim excess Coe-Flo with periphery of base plate with warm, sharp knife. Stabilized base plate should be kept in cold water.

3737 WEST 127TH STREET
ALSIP, IL 60803