Prior to use, carefully read the instructions for use.

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DE

COE-PAK™
COE-PAK AUTOMIX NDS™
PERIODONTAL DRESSING
NO EUGENOL, NO ASBESTOS

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS:
For surgical dressing and periodontal pack.
Use Coe-Pak whenever a periodontal paste is required.

CONTRAINDICATIONS:
In rare cases this material may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE:
1. a) For non-automix, use a spatula and a COE mixing pad. Do not use a rigid or narrow spatula. Thorough mixing is very important. This is more easily done with a wide, flexible spatula.
b) When using COE-PAK AUTOMIX NDS, load the cartridge into the GC Cartridge Dispenser II and attach a mixing tip. Express the required length of COE-PAK onto the mixing pad. (Do not use more than 4 full strokes of the trigger). Leave the AUTOMIX NDS mixing tip in place until required for the next use.
2. For non-automix, extrude equal lengths of paste from catalyst and base tubes. Spatulate for 30-45 seconds or until the two pastes are a uniform color. If you would like a harder dressing, add a small amount of zinc oxide powder to the pink paste (catalyst) before spatulating.
3. Using a spatula, form the mixed pastes into a cylindrical shape.
4. Lightly lubricate your fingers with a thin film of vaseline, lanolin, cold cream, or water and test the mixed pastes for tackiness (it should lose its tackiness after 2-3 minutes; 1 minute for Hard & Fast set). NOTE: COE-PAK AUTOMIX NDS can be handled after about 30 seconds. Lubrication of fingers is not necessary.
5. COE-PAK can be handled and molded 2 to 3 minutes after mixing. With your fingertips, roll the cylindrical form off the COE mixing pad (1 minute for Hard & Fast).
6. COE-PAK remains workable for 10-15 minutes (5-8 minutes for Hard & Fast and at least 8 minutes for Automix NDS). During this time it can be formed in the usual manner into ropes or other desired shapes.
7. During its working stage COE-PAK is extremely cohesive. It can be formed into ropes of virtually any diameter for easy placement against tissue and teeth.

COMPARISON OF COE-PAK/COE-PAK HARD & FAST/COE-PAK AUTOMIX NDS:
(equal lengths -1/2 minute mixing time)

	COE-PAK REGULAR	COE-PAK HARD & FAST	COE-PAK AUTOMIX NDS
After mixing, ready to use	3 minutes	1 minute	30 seconds
Consistency	plastic	heavy, but still plastic	plastic
Working time	10-15 minutes	5-8 minutes	at least 8 minutes
Ability to bond one piece of material to another	10-15 minutes	5-8 minutes	8-12 minutes
Final hardness	30 minutes (still resilient but not brittle)	10 minutes (much harder than COE-PAK, but not brittle)	10 minutes

CAUTION:

- Sensitivity to some of the components of this product has been reported. The reported sensitivity varies from simple dermal redness, to rash, swelling and nausea and/or vomiting. The practitioner should assure, prior to use, no such sensitivity to this product is present in the intended patient.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In case of contact with skin, wash with soap and water. If irritation develops, seek medical attention.

NOTE: GC Cartridge Dispenser II is suggested for use with the NDS, New Delivery System cartridge.

STORAGE:

Store COE-PAK in a dry location at normal room temperature (70 to 77F; 21 to 25C). The maximum shelf life of these products is 24 months from the manufacturing date.

PACKAGES:

- Tubes
- COE-PAK Standard (or Regular) Package 90g tube base, 90g tube catalyst
 - COE-PAK Hard and Fast Package 90g tube base, 90g tube catalyst
- Cartridges
- 2-Pack COE-PAK AUTOMIX NDS Cartridges 2 cartridges (48 mL each), 12 mixing tips

Last revised: 11/2011

CE 0086

MANUFACTURED by:
GC AMERICA INC.
3737 WEST 127th STREET, ALSIP IL 60803 U.S.A.
TEL:+1-708-597-0900

DISTRIBUTED by:
EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00
PRINTED IN U.S.A.

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