

COE CapMixer™

High Speed Mixer

for Encapsulated Dental Materials

Operating Instructions:

The COE CapMixer is a high quality touch pad mixer and is capable of mixing all modern encapsulated dental products. Its mixing speed is 4400 rpm and time can be varied from 0 – 30 seconds in 1 second intervals.

There is a power switch in the back of the machine and a 4 amp fuse. When the machine is turned on, it automatically sets itself to 10 seconds. Adjust the mixing time if necessary to optimize extrusion amount. The left time button (↑) is pressed to increase time up to 30 seconds; the right time button (↓) is pressed to decrease time. The on/off button is pressed to activate or stop the machine. To avoid accidental operation, all buttons are programmed to operate only when pressed and held 1/2 second.

Open the protective cover and seat the capsule in the capsule holder. Make sure the capsule is inserted with the tip facing to the left, and the right side of the capsule is resting flat against the right side of the capsule holder. Close the cover and press the on/off button for the automatic 10 second mix or adjust the time according to the instructions above.

After mixing is completed remove the capsule from the unit.

Cleaning Instructions:

All external surfaces may be cleaned by wiping with a soft cloth moistened with disinfectants or isopropyl alcohol. CAUTION: Use of petroleum based solvents, iodophors or phenolic based products is contraindicated, in that, they may react with the mixers surface.

UL/CSA Approved

Specifications:

- 120 Volts, 50-60 Hz
- 155 Watt, 4400 rpm
- 4 amp fuse
- Dimensions: 5h x 7.5w x 8d inches
- Weight - 5.9 pounds (actual unit)



Distributed by:
GC America Inc.
3737 W. 127th Street
Alsip, IL 60803

Tel: 1-708-597-0900