

Disinfects the following organisms on hard nonporous inanimate environmental surfaces: Herpes Simplex Type 1 virus, Herpes Simplex Type 2 (genital) virus, Influenza A₂ (Hong Kong), Human immunodeficiency virus HIV-1 (AIDS virus), Staphylococcus aureus, Pseudomonas aeruginosa, Salmonella choleraesuis, & Human Coronavirus (HCoV).

Used on: Counter tops, table tops, plastic surfaces, sinks, bath tubs, wash basins, formica, exhaust fans, stainless steel, finished woodwork, glazed porcelain, leatherette, plastic.

Used in: Hospitals, institutions, office buildings, factories, motels, and meat and poultry plants. Leaves the surface clean and free from odors. Excellent for all bathroom fixtures, grouting tile, and shower stalls.

Also excellent for cleaning vinyl, leather, and plastic upholstery, varnished, painted and enameled surfaces. Contains no abrasives.

DIRECTIONS FOR USE

It is a violation of federal law to use this product in a manner inconsistent with its labeling.

To clean: Remove cap, shake, spray evenly over the surface, wipe clean. For heavily soiled surfaces, allow foam to remain for a minute before wiping.

To disinfect: Clean surface thoroughly as directed above. Then spray a thin coat of foam, making sure it covers the entire surface to be disinfected. Allow to remain for at least 10 minutes, then wipe off with clean damp cloth. Rinse all surfaces that come in contact with food such as food processing equipment, tables, counters, appliances with potable water. Do not use on utensils, dishware, or cookware.

Food Processing Plant Directions: Before using this product, food products and packaging materials must be removed or carefully protected. Use on floors, walls, tables, or other hard nonporous surfaces. Follow the above disinfection directions.

KILLS HIV-1 ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (associated with AIDS).

SPECIAL INSTRUCTION FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 (AIDS VIRUS) OF SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS.

PERSONAL PROTECTION:

Disposable gloves should be worn when cleaning blood, body fluids, or contaminated items. Hands should be washed immediately and thoroughly after gloves are removed, or if hands are contaminated with blood or body fluids. Masks

and protective eyewear or faceshields should be worn during cleaning procedures involving blood or body fluids. Gowns or aprons should be worn during cleaning procedures for protection against splashes of blood or other body fluids.

CLEANING PROCEDURE: Blood and other body fluids must be thoroughly cleaned from surfaces and objects with a suitable detergent before application of the disinfectant. After cleaning is complete, follow disinfection directions described previously.

CONTACT TIME: Allow at least 10 minutes contact time with the surfaces/objects for disinfection.

DISPOSAL OF INFECTIOUS MATERIALS: Contaminated waste and blood and other body fluids should be autoclaved and disposed according to local regulations for infectious waste disposal.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body.

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals
WARNING: Causes substantial but temporary eye injury. Harmful if absorbed through the skin. Avoid breathing spray mist. Do not get in eyes, on skin, or on clothing. Wear safety glasses or goggles. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco. Remove and wash contaminated clothing before reuse.

PHYSICAL OR CHEMICAL HAZARDS

Contents under pressure. Do not use or store near heat or open flame. Do not puncture or incinerate container. Exposure to temperatures above 120°F may cause bursting.

STORAGE AND DISPOSAL

STORAGE: Store in a cool dry area away from heat or open flame. **DISPOSAL:** Do Not puncture or incinerate!

If empty: Place in trash or offer for recycling if available. **If partly filled:** Call your local solid waste agency or 1-800-CLEANUP for disposal instructions.

Buyer assumes all risk of use, storage or handling of this product not in strict accordance with directions given herewith.

For Commercial/
Institutional Use Only

EPA Reg. No. 11694-34-10214
EPA Est. No. 11694-KS-1



COE FOAM™ II

GERMICIDAL CLEANER/ SURFACE DEODORANT

REORDER NO. 551222

CLEANER - DISINFECTANT - DEODORIZER

Disinfects Herpes Simplex Virus
Type 1 & 2, Influenza A2
(Hong Kong) Virus,
Human Coronavirus (HCoV) & HIV-1 (AIDS virus)±

SPRAY
AND
WIPE

±Kills HIV-1 (AIDS virus) on precleaned environmental surfaces/objects previously soiled with blood/body fluids.

ACTIVE INGREDIENTS:

n-Alkyl (60% C₁₆, 30% C₁₂, 5% C₁₄, 5% C₁₈)
dimethyl benzyl ammonium chlorides .0.1%
n-Alkyl (68% C₁₂, 32% C₁₄) dimethyl
ethylbenzyl ammonium chlorides0.1%

INERT INGREDIENTS99.8%
TOTAL100.0%

KEEP OUT OF REACH OF CHILDREN WARNING

FIRST AID: Have the product container or label with you when calling a Poison Control Center or doctor, or going for treatment. **IF IN EYES**

- Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice. **IF INHALED** - Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration preferably mouth-to-mouth if possible. Call a poison control center or doctor for further treatment advice. **IF SWALLOWED** - Call a poison control center or doctor immediately for treatment advice. Do not induce vomiting unless told to do so by a poison control center or doctor. Do not give anything by mouth to an unconscious person. **IF ON SKIN OR CLOTHING** - Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice.

See back panel for additional precautionary statements.

NET WT. 18 OZ
(1lb 2 oz) (510 g)

Manufactured for:

GC
GC America Inc.
Alsip, IL 60803