Prior to use, carefully read the instructions for use.

EN

## CAVITY CONDITIONER

CAVITY CLEANING AGENT

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

CAVITY CONDITIONER is a mild polyacrylic acid solution designed to remove the smear layer and to condition dentine and enamel, thus enhancing the bond between glass ionomer cement and tooth structure.

### CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

- After tooth preparation, apply CAVITY CONDITIONER to the bonding surfaces for 10 seconds using a cotton pellet or sponge.
- Rinse thoroughly with water and dry. DO NOT DESICCATE.
- Apply GC Fuji BOND LC, GC Fuji II LC (improved) or other glass ionomer as a bonding agent, base/liner or restorative material.
- Close the bottle immediately after use.

### STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life : 3 years from date of manufacture)

### PACKAGES

Bottle of 6g (5.7mL).

### CAUTION

- In case of contact with oral tissue or skin, flush immediately with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

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