CAVITY CONDITIONER

CAVITY CLEANING AGENT

Prior to use, carefully read
the instructions for use.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

GC CAVITY CONDITIONER is a mild polyacrylic acid solution designed to remove the smear layer and to condition dentine and enamel, thus enhancing the bond between glass ionomer cement and tooth structure.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

1. After tooth preparation, apply GC CAVITY CONDITIONER to the bonding surfaces for 10 seconds using a cotton pellet or sponge.
2. Rinse thoroughly with water and dry. DO NOT DESICCATE.
3. Apply GC Fuji Bond LC, GC Fuji II LC (improved) or other glass ionomer as a bonding agent, base/liner or restorative material.
4. Close the bottle immediately after use.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).
(Shelf life : 3 years)

PACKAGES

Bottle of 6g (5.7mL).

CAUTION

1. In case of contact with oral tissue or skin, flush immediately with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.