Prior to use, carefully read the instructions for use.

FIT CHECKER

SILICONE FOR REVEALING HIGH SPOTS AND PRESSURE POINTS

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

FIT CHECKER is a most suitable material for revealing high spots and pressure points of a prosthesis. All procedures can be done quickly, easily and neatly.

To check the fit of

*Crown : margin, proximal surface, occlusal surface.

*Precast post : fit of post.

*Inlay : margin, proximal surface.

*Denture : mucosal surface.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

- Mixing
 - Squeeze out equal lengths of base and catalyst (1 cm for crown or inlay, 2-3 cm for partial denture). Mix thoroughly for 20 seconds.
- Setting
 - Apply the mixture to the prosthesis and set it into the mouth. Have the patient bite slowly in the centric occlusal position for 1.5 minutes.
- Assessment
 - Remove the prosthesis carefully with a suitable instrument and check it. A suitable fit is shown by being uniformly covered with a thin film (30-120 µm) : if there is a spot where the film appears to be extremely thin or thick, it should be corrected using standard techniques.
- Removing and cleaning
 - After assessment, remove the material and clean the prosthesis.

Notes

- When the room temperature is high, use the retarder to prolong the working time. (Add drop before mixing).
- Do not mix components of other silicone impression materials.

COLOURS

White, Black

STORAGE

Store at normal room temperature (23°C / 73.4°F). (Shelf life : 2 years from date of manufacture)

PACKAGES

1-1Pkg : 80g (64mL) base, 20g (20mL) catalyst, 3g retarder and mixing pads.

CAUTION

- In case of contact with oral tissue or skin, remove and flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.