Prior to use, carefully read the instructions for use.

EN

## FREEGENOL TEMPORARY PACK

EUGENOL FREE TEMPORARY CEMENT

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

Cementation of temporary crowns and bridges.

### CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

1. Squeeze equal lengths of Base and Accelerator pastes onto the mixing pad.
2. Mix thoroughly for 20-30 seconds.
3. Insert paste into (temporary) crown or bridge and seat onto the prepared dry tooth.
4. Allow to set (3 minutes from start of mixing) and trim excess.

Note:

1. Setting time can be adjusted by using more or less Base (More Base-faster set ; less Base-slower set). Do not go outside Base / Accelerator ratio range 3 : 2 to 2 : 3).
2. If material sticks to tooth or metal, Vaseline may be added up to 25% of total mix.

### STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F). (Shelf life: 3 years from date of manufacture)

### PACKAGES

1-1 pkg : 55g Base, 20g Accelerator, Cleaner and mixing pad.

### CAUTION

In case of contact with eyes, flush immediately with water and seek medical attention.

Some products referenced in the present IFU may be classified as hazardous according to GHS.

Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com/download> or for The Americas <http://www.gcamerica.com> They can also be obtained from your supplier.

Last revised : 07/2015

CE 0086

'GC'

MANUFACTURED by  
GC DENTAL PRODUCTS CORP.  
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan  
TEL: +1-708-597-0900.

RESPONSIBLE MANUFACTURER IN CANADA:  
GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900.

GC ASIA DENTAL PTE. LTD.  
11 Tampines Concourse, #03-05, Singapore 528729  
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.  
1753 Botany Rd, Bankmeadow, NSW 2019, Australia  
TEL: +61 2 9316 4499

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