EN
Prior to use, carefully read the instructions for use.

G-CEM LinkForce™ ADHESIVE RESIN CEMENT G-PRIMO BOND DCA DUAL CURE ACTIVATOR FOR G-PRIMO BOND

G-CEM™ Try-In Paste TRY-IN PASTE FOR G-CEM LinkForce™ AND G-CEM™ Veneer

For use only by a dental professional in the recommended conditions.

RECOMMENDATIONS Indications of all types of all ceramic, resin and metal-based inlays, crowns and bridges.

2. Cementation of orthodontic ceramic fiber posts, cast post and cores.

3. Cementation of all ceramic and composite veneers.

4. Temporary cementation of cast or preformed on implant abutments and veneers.

5. Temporary RESTORATION
a) Remove the temporary restoration and clean the prepared tooth using a brush and slurry of pumice to remove the temporary cement.

6. Check the fit, shade and occlusion of the final restoration. If necessary, take new impressions with G-CEM™ Try-In Paste for trial fit.

7. Hold, spray uniformly and remove wing cap by turning counterclockwise.

8. Promote fully adhesive attaching dispensing tip to the syringe by turning clockwise.

9. Rinse the preparation thoroughly and clean the prepared tooth using a brush and slurry of pumice to remove the temporary cement.

10. Check fit, shade and occlusion of the final restoration. Disperse thoroughly and clean the surface of the restoration and spot on preparation.

11. Place a release cap on promptly after use.

12. Avoid handling the restoration with fingers. Remove restoration and thoroughly rinse G-CEM™ Try-In Paste with water. Take new impressions with G-CEM™ Try-In Paste as residual paste causes decrease in bond strength.

13. PRE-TREATMENT OF THE RESTORATION
The restoration is pretreated and handled according to the manufacturer's instructions.

• For glass ceramics, etch with hydrofluoric acid. Rinse thoroughly with water and dry.

• For zirconia, alumina, alumina hybrid ceramics and flow composites, sandblast with glass beads and clean with an oil-free air syringe. If necessary, clean with an ultrasonic spray. Rinse thoroughly with water.

• For metal, etch with phosphoric acid. Rinse thoroughly with water. Follow manufacturer's instructions for surface parameters (sandblasting and acid etching).

• For CERAMSTAR™: If sandblaster is not available, etch with phosphoric acid for 30 seconds.

• Apply G-Multi PRIMER to the bonding surface and dry thoroughly.

• PRE-TREATMENT OF THE PREPARATION
a) Clean the preparation with pumice and water. Dry by gently blowing with compressed air.

b) Apply calcium hydroxide to the preparation, select from the following three techniques.

Self-etch technique
Apply G-Multi PRIMER to cut-but enamel and dentin, followed by a separate etching step.

Note:
Cut-but enamel should always be treated with a 35-40% phosphoric acid gel (e.g. GC ETCHANCH) for 10 seconds, followed by water for 30 seconds and gently dried.

Selective etching of enamel
Prior to applying with G-PRIMO BOND to enamel and dentin, etch only enamel with 35-40% phosphoric acid gel (e.g. GC ETCHANCH) for 10 seconds, followed by water for 30 seconds and gently dried.

Total-etch technique
Apply G-PRIMO BOND to enamel and dentin, etch enamel and dentin with 35-40% phosphoric acid gel (e.g. GC ETCHANCH) for 10-15 seconds, rinse for 5 seconds and gently dry.

NOTE:
In case of veneer cementation, select either selective etching or total etch for composite, metal core, roughen the surface with a diamond bur and proceed directly to the application of G-PRIMO BOND.

Application of G-PRIMO BOND
Apply G-PRIMO BOND composite, metal abutment) should be used. Apply G-PRIMO BOND to the prepared tooth. Leave undisturbed for 5 seconds. Using an air syringe dry the surface for 5 seconds.

13. In case of veneer cementation, G-PRIMO BOND should always be used without DCA and light cured.

14. DISPENSING
a) Dispensing: Pressing the first time from a new syringe or after a long interval in the syringe, bleed the paste in the syringe.

b) Remove cap of the syringe by rotating 1/4 turn counterclockwise. Attach a GC Activator Tip by aligning the notch with the notch on the syringe. Use the V-shaped notch between the syringe barrels. Pump firmly until the mixing is complete. Leave the colored color of the mixing tip 1/4 turn clockwise.

c) Extrude material directly into restoration.

Used mixing tip can be left on the syringe to serve as a storage cap until next use. For each application, provide a new mixing tip.

15. CEMENTATION
a) Apply G-PRIMO BOND to the surface of the restoration with sufficient cement and seal immediately. The working time is 3 minutes. The start of the setting is 2 minutes. Higher temperatures will shorten the working time.

b) Maintain moderate pressure.

16. FINISHING
a) Ambient light may shorten the working time. It is recommended to use a curing light during the cementation procedure.

2. In case of veneer cementation, it is recommended to use a curing light for 2 minutes. Higher temperatures will shorten the working time.

17. EXCESS CEMENT REMOVAL
a) Maintain moderate pressure and make sure the restoration remains in place.

b) Clean surface with alcohol. Use a visible light curing unit for 1-2 seconds or allow the cement to self-cure for 2 minutes.

c) Remove excess cement with appropriate instruments, avoiding the restoration in position.

18. HOLDING THE RESTORATION IN POSITION
While maintaining moderate pressure, light cure each surface of the restoration for 2 minutes (4000 mW/cm²). If the light application is difficult, let it set for 4 minutes after seating the restoration. For veneer cementation, place the restoration in position for 4 minutes.

19. FINAL POLISHING AND ADJUSTMENTS
a) Use fine sand and polish margins with appropriate instruments.

Be careful not to overheat.

20. CEMENTATION TECHNIQUE for metal, ceramic, fiber posts, and cast post and cores.

1. PRE-TREATMENT OF THE POST
Metal posts that post is pretreated and handled according to manufacturer's instructions.

2. PRE-TREATMENT OF THE BONDING SURFACE AND DRY
Apply G-Multi PRIMER to the bonding surface and dry thoroughly. Refer to G-Multi PRIMER instructions.

3. DISPENSING & CEMENTATION
a) Dispense moderate pressure making sure the restoration remains in place.

b) Ambient light may shorten the working time. It is recommended to use a curing light during the cementation procedure.

3. When the cement is in contact with G-PRIMO BOND DCA, accelerate the setting by curing for 4 minutes.

4. Keep the tip immersed in the cement while dispensing to avoid dripping.

5. DO NOT use linted spirals to place the cement in the post space as this may shorten the working time.

6. Use a storage cap until next use. For each application, provide a new storage cap.

7. EXCESS CEMENT REMOVAL
a) Excise moderate pressure making sure the post remains in place.

b) Clean surface with alcohol. Use a visible light curing unit for 1-2 seconds or allow the cement to self-cure for 2 minutes.

19. FINAL SET
a) While maintaining moderate pressure, light cure each surface of the restoration for 4 min. after inserting post.

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