



*Editors' Choice*  
THE DENTAL ADVISOR  
www.dentaladvisor.com

RATINGS:  
Excellent + + + + +  
Very Good + + + +  
Good + + +

## G-Premio Bond™

*GC America, Inc*  
www.gcamerica.com

### Description

**G-Premio Bond™** is a universal (8th-generation), single component, light-cured bonding agent. It is compatible with total-etch, self-etch and selective-etch techniques and indicated for use in all classes of direct restorations. The addition of MDP monomer enhances chemical adhesion to zirconia, alumina and other non-precious metals; increased water content improves removal of the smear layer; and increasing the amount of carboxylic monomer maintains an acidic pH. **G-Premio Bond** is recommended for use in the following indications: direct bonding of light-cured composites and compomers to tooth structure; intraoral repair of composite; metal-based and zirconia/alumina-based restorations, in combination with **G-CEM LinkAce™** resin cement for the bonding of indirect restoration where light curing is possible; treatment of hypersensitive teeth; and sealing of tooth preparation (cavity or abutment) for indirect restorations. **G-Premio Bond** can also be used in combination with a silane coupler when repairing glass or hybrid ceramic. It is ideal for use in indirect indications with its very low film thickness (3 µm). It is available in a 5 mL bottle with a new silicone cover to enable more precise drop-by-drop dispensing to eliminate waste and to insulate the bottle from heat generated in the hands. It is also available in 50-count unit-dose packs (0.1 mL each). Curing times is 10 seconds for most lights and 5 seconds for lights over 1200 mW/cm<sup>2</sup>. **G-Premio Bond unit dose** was evaluated by 39 consultants in 1131 uses. This universal bonding agent received a 96% clinical rating.



### Consultants' Comments

*"I like the unit-dose container; it stands nicely on the tray and the lid closes securely."*

*"No problems with post-op sensitivity during the evaluation."*

*"Can be stored at room temperature."*

*"The foil wrappers keep the liquid fresh and the material from drying out."*

*"Make the unit-dose container shallower to accommodate the bend in the brush."*

### Suggested Retail Cost

**G-Premio Bond** \$138.50/5 mL bottle


**G-Premio Bond** \$163.50/50 unit dose

## Product Features


**G-Premio Bond** is a universal bonding agents that provides a quick technique, with a 10-second soak with no scrubbing required. It is compatible with all etching techniques and can be used with dual-cured composites, provided that light curing is initiated. The unit dose container has a wide enough opening to accommodate all brushes, and the cap can be closed to eliminate evaporation of the solvent during the procedure. The viscosity of G-Premio Bond wets the tooth well, and the acetone solvent dissipates quickly with air. Overall, **G-Premio Bond** is easy to use and effective for adhesive restorative procedures.

## Clinical Tip

- Leave bond undisturbed for 10 seconds after application; dry for 5 seconds.

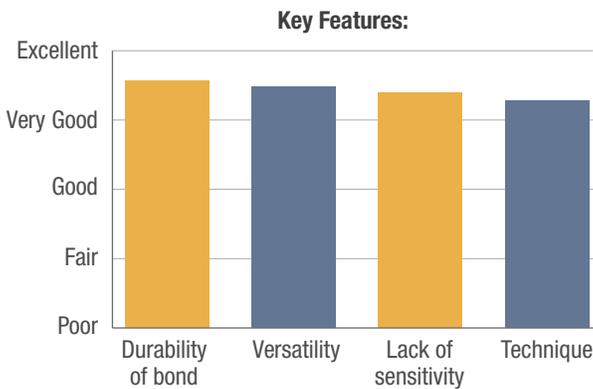

**Compared to Competitive Products:**

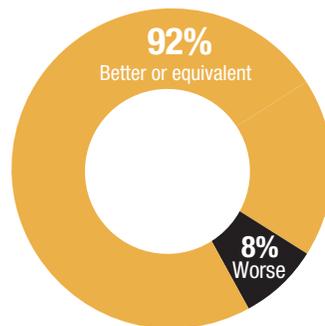

**Percentage of Consultants Who Would:**

