




G-Premio BOND™

ONE COMPONENT LIGHT-CURED ADHESIVE

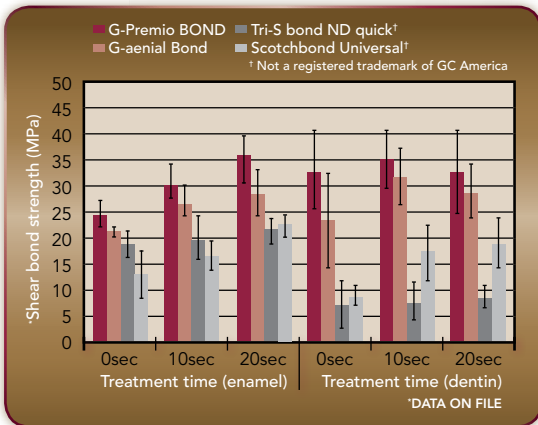

Self-etch courtesy of Dr. Iбата, Japan



Selective-etch courtesy of Dr. Lassocinski, Poland



Total-etch courtesy of Dr. Browet, Belgium

TOUCH of GENIUS™

- Flexible for use in total, selective and self-etch bonding techniques
- Fast application (10 seconds)
- Good stability with extended working time (up to 5 minutes)
- New bottle design for better dispensing control

Fast & Consistent Universal Bonding



Fast and Easy Application



	DISTANCE FROM LIGHT GUIDE TIP	
	< 10 mm	> 10 mm
Halogen/LED (700 mW/cm ²)	10 sec	20 sec
High power LED e.g. G-Light (more than 1200 mW/cm ²)	5 sec	10 sec

Many changes to create a better bonding agent

The dispersion of silica filler was improved and viscosity was reduced to increase working time by 333%. The amount of water was increased to dissolve and remove smear layer better and the amount of carboxylic monomer was increased to keep acidic pH. The amount of photoinitiator was also increased to make a firm bonding layer. These adjustments

all result in a faster application and shorter treatment time. The new bottle contains a unique, silicone cover. This redesign offers many benefits such as a more precise drop by drop dispensing to eliminate waste and keeps heat from fingers from impacting bonding performance by serving to insulate the bonding agent.

UNIVERSAL BOND	INDICATION		FEATURES
		Direct Restoration	
	Indirect Restoration		Bonding to enamel and dentin Thin film thickness (3µm) allows light cure technique
		+ Dual Cure Activator	Can be made dual cure by mixing with Dual Cure Activator
		+ Multi primer	Reliable primer for all substrates
	Densitizer		Can be used for hypersensitivity treatment

009275 G-Premio BOND Kit
contains: 1 bottle (5 mL), 50 disposable applicators, 20 trays

009276 G-Premio BOND Unit Dose
contains: 50 unit doses (0.1 mL)

009277 G-Premio BOND Bottle Refill
contains: 1 bottle (5 mL)

