# G-ænial™ Bond Unit Dose

ONE COMPONENT SELF-ETCHING LIGHT-CURED ADHESIVE

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

- Bonding of light cured composites and acid modified composites (compomers) to tooth structure.
- Bonding of dual cured luting and core build up composites to tooth structure as long as these materials are light cured.

### CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### NOT TO BE USED

- In combination with chemically cured composite resin.
- In combination with eugenol containing materials as eugenol may hinder G-ænial Bond from setting or bonding properly.
- In combination with desensitizers, as desensitizers may hinder G-ænial Bond from setting or bonding properly.
- In combination with dual cured luting and core build up composites unless both G-ænial Bond and these materials are individually light cured.

### DIRECTIONS FOR USE

#### 1. CAVITY PREPARATION

Prepare tooth using standard techniques. Make sure that the temporary material is thoroughly removed from the surface. Use rubber dam for protection. Dry the prepared tooth surface by gently blowing with an air syringe.

#### 2. TECHNIQUE SELECTION

- Select out of the following 2 technique options
- Self-etch technique – apply G-ænial Bond to bur cut enamel and dentin, without a separate etching step.
  - Selective etching of enamel – prior to applying G-ænial Bond to enamel and dentin, etch (bur cut) enamel with 35-40% phosphoric acid gel for 10 seconds, rinse for 5 seconds and gently dry.

Note: Non bur cut enamel should always be treated with a 35-40% phosphoric acid gel for 10 seconds, rinsed with water for 5 seconds and gently dried.

#### 3. APPLICATION

- Unscrew cap counterclockwise (Fig. 1).
- Using the disposable applicator provided, mix the liquid thoroughly for 5 seconds as illustrated (Fig. 2), making sure to combine liquid from top and bottom of the container.
- IMMEDIATELY apply to the prepared enamel and dentin surfaces using the disposable applicator (Fig. 3).
- Leave undisturbed for 10 seconds after the end of application (Fig. 4).
- Then, dry thoroughly for 5 seconds with oil free air under MAXIMUM air pressure. Use vacuum suction to prevent splatter of the adhesive (Fig. 5). The final result should be a thin adhesive film with the appearance of frosted glass and which does not visibly move under further air pressure.

#### 4. LIGHT CURING

Light cure using a visible light curing unit (Fig. 6). Irradiation time:

Halogen / LED (700 mW/cm <sup>2</sup> )	: 10 seconds
Plasma Arc (2000 mW/cm <sup>2</sup> )	: 3 seconds
G-Light (1200 mW/cm <sup>2</sup> )	: 5 seconds

In case where the light guide tip is more than 10mm away from the surface to be irradiated, light cure for the following time:

Halogen / LED (700 mW/cm <sup>2</sup> )	: 20 seconds
Plasma Arc (2000 mW/cm <sup>2</sup> )	: 6 seconds
G-Light (1200 mW/cm <sup>2</sup> )	: 10 seconds

- When removing from refrigerator after long-term storage, leave the G-ænial Bond Unit Dose to stand at room temperature for several minutes prior to use.
- When opening the cap, make sure to identify the top and bottom of the unit dose capsule.
- If the cap of the unit dose is too tight to open, do not try to open it with force. Use another unit dose.
- Apply G-ænial Bond immediately as the material contains a volatile solvent.
- Remove excess material remaining on the tooth (other than the surfaces to be bonded) using a sponge or cotton pellet as residue material is hard to remove after light curing.
- Should the applied material be contaminated with water, blood or saliva prior to light curing, wash and dry the tooth and repeat the procedure by re-applying material.

### 5A. PLACEMENT OF LIGHT CURED COMPOSITES AND COMPOMERS

After having light cured the adhesive, follow manufacturer's instructions for use in order to place, contour and light cure the composite resin material of choice.

### 5B. PLACEMENT OF DUAL CURED COMPOSITES

After having light cured the adhesive, make sure to light cure the dual cured composite material separately. Self curing only will result in insufficient bonding strength.

### 6. FINISHING

Adjust restoration, finish and polish using standard techniques.

### STORAGE

Store at room temperature (1-28°C) (33.8-82.4°F). If not in use for a prolonged period of time, store in refrigerator. Shelf life: 2 years from date of manufacture.

### PACKAGE

G-ænial Bond Unit Dose 0.1mL (50), disposable applicator (fine) (50)

### CAUTION

- G-ænial Bond is flammable. Do not use near naked flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from direct sunlight.
- G-ænial Bond is volatile. Use in a well ventilated place. Advise patients to breathe through nose.
- This material is disposable. Do not recap or reuse as this may cause decrease of bonding strength and contamination.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In case of contact with oral tissue or skin, immediately remove with a sponge or cotton pellet. After the restorative treatment is finished, rinse thoroughly with water.
- If the tissue contacted by the material turns white or forms a blister, advise the patient to leave the affected area undisturbed, until the mark disappears, usually in 1-2 weeks. To avoid contact, it is recommended to apply cocoa butter to the area where rubber dam cannot cover.
- Avoid inhalation or digestion of material.
- When spilled on the table or floor, immediately wipe off with a dry cloth.
- Do not mix with other products.
- Dispose of all waste according to local regulations.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available for the Americas at: <http://www.gcamerica.com>. They can also be obtained from your supplier.

Last revised : 08/2016

**CE 0086**

**GC**

MANUFACTURED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan  
RESPONSIBLE MANUFACTURER IN CANADA:  
GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
DISTRIBUTED by  
GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900  
PRINTED IN JAPAN

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