Prior to use, carefully read the instructions for use.

EN

G-ænial Sculpt™

LIGHT-CURED UNIVERSAL NANO HYBRID COMPACTABLE COMPOSITE

For use only by a dental professional in the recommended indications.

INDICATIONS

- Direct restorative for class I, II, III, IV and V cavities.
- Direct restorative for wedge-shaped defects and root surface cavities.
- Direct restorative for veneers and diastema closure

CONTRAINDICATIONS

- Direct pulp capping.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.


DIRECTIONS FOR USE

- Shade Selection
Clean the tooth with pumice and water. Shade selection should be made prior to isolation.
Select the appropriate shade from 17 shades of A1, A2, A3, A3.5, A4, B1, B2, B3, C3, AO1, AO2, AO3, BW, XBW, CV, JE, AE.
A, B, C shades are based on the Vita® classical shade guide.
*Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany
- Cavity Preparation
Prepare cavity using standard techniques. Dry by gently blowing with oil free air. Rubber dam is recommended to isolate prepared tooth from contamination with saliva, blood or sulcus fluid.
Note:
For pulp capping, use calcium hydroxide.
- Bonding Treatment
For bonding G-ænial Sculpt to enamel and/or dentin, use a light-cured bonding system such as G-Premio BOND, GC G-ænial Bond, GC G-BOND, GC Fuji Bond LC, GC UniFil® Bond. Follow manufacturer's instructions.
- Placement of G-ænial Sculpt
 - Dispensing from a Unitip
Insert the G-ænial Sculpt Unitip into a commercially available applicator. Refer to the applicator manufacturer's instructions for use. Remove the cap and extrude material directly into the prepared cavity. Use steady pressure.
 - Dispensing from a syringe
Remove syringe cap and dispense material onto a mixing pad. Place the material into the cavity using a suitable placement instrument. After dispensing, screw plunger anticlockwise by a half to full turn to release residual pressure inside the syringe. Replace cap immediately after use.
Note:
 - Material can be applied in a single shade layer to achieve aesthetic restorations using Universal shades. For details, refer to the Clinical Hints.
 - Material may be hard to extrude immediately after removing from cold storage. Prior to use, let stand for a few minutes at normal room temperature.
 - After dispensing, minimize exposure to ambient light. Ambient light can shorten the manipulation time.

Clinical Hints

- In the case of small cavities
Restore using a one shade technique. In most cases the use of one Universal shade alone will be sufficient. In cases where a higher degree of translucency is needed, one of the Enamel shades can be selected.
- In the case of large and/or deep cavities
In most cases a multi shade layering technique will give the best aesthetic results. To block out shine through from the oral cavity or to mask discolored dentin, select an appropriate Opaque shade and continue to build up with a Universal shade. For optimal esthetics use an Enamel shade as the final composite layer.
In the case of deep posterior cavities, a flowable composite such as G-ænial Flo, GRADIA DIRECT Flo / LoFlo (GC) or a glass ionomer cement such as GC Fuji LINING LC (Paste Pak) or GC Fuji IX GP can be used on the cavity floor instead of an opaque shade.

Examples of Clinical Applications (Clinical Hint)

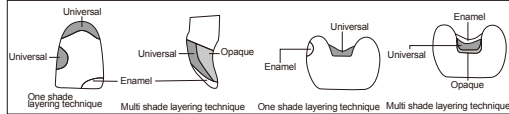5. CONTOURING BEFORE LIGHT CURING

Contour using standard technique.

6. LIGHT CURING

Light cure G-ænial Sculpt using a light curing unit. Keep light guide as close as possible to the surface. Refer to the following chart for Irradiation Time and Effective Depth of Cure.

Irradiation Time and Effective Depth of Cure	9 sec. (Plasma arc) (2000mW/cm ²)
	10 sec. (LED Light) (1200mW/cm ²) 20 sec. (Halogen/LED) (700 mW/cm ²)
AO1, AO2, AO3, A3.5, A4, CV	1.5mm
A1, A2, A3, B1, B2, B3, C3, BW, XBW	2.0mm
JE, AE	2.5mm

- Note:
- Material should be placed and light cured in layers. For maximum layer thickness, please consult above table.
 - Lower light intensity may cause insufficient curing or discoloration of the material.

7. Finishing and Polishing

Finish and polish using standard techniques.

SHADES

A1, A2, A3, A3.5, A4, B1, B2, B3, C3, AO1, AO2, AO3, XBW, BW, CV, JE, AE

STORAGE

Recommended for optimal performance, store in a cool and dark place (4 - 25°C / 39.2 - 77.0°F)
(Shelf life : 3 years from date of manufacture)

PACKAGES

- Unitips
 - Pack of 20 tips (each in 9 shades) (0.16mL per tip)
A1, A2, A3, A3.5, A4, B1, B2, B3, C3
 - Pack of 10 tips (each in 8 shades) (0.16mL per tip)
AO1, AO2, AO3, XBW, BW, CV, JE, AE
NOTE :
Weight per unitip : 0.3g
- Syringes
 - Syringe (in 17 shades) (2.0mL per syringe)
NOTE :
Weight per syringe : 3.8g

CAUTION

- In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take care to avoid ingestion of the material.
- Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
- For infection control reasons, Unitips are for single use only.
- The barrel of the dental composite applicator needs to be disinfected between patient uses. Please use an alcohol wipe.
- Wear protective eye glasses during light curing.
- When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
- Do not mix with other similar products.
- Avoid getting material on clothing.
- In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge, or cotton pellet before light curing.
- Do not use G-ænial Sculpt in combination with eugenol containing materials as eugenol may hinder G-ænial Sculpt from setting.
- Dispose of all wastes according to local regulations.

Last revised: 04/2015

GC R_xOnly

MANUFACTURED by
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

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