


G-ænial Sculpt®

LIGHT-CURED UNIVERSAL NANO
HYBRID COMPACTABLE COMPOSITE

THE UNIVERSAL
COMPOSITE
THAT KEEPS
POLISHING IN MOUTH



SELF POLISHING/SHINING EFFECT

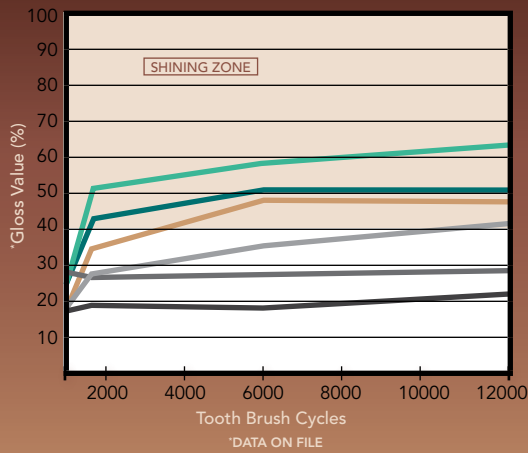


	G-ænial Sculpt		Filtek Supreme Ultra†
	G-ænial Flo		Estelite Sigma Quick†
	G-ænial Universal Flo		Majesty ES-2†

† Not a registered trademark of GC America


TOUCH of GENIUS™

- Superior polish and high gloss retention
- Self-polishing effect enables additional polish in mouth
- Extraordinary wear resistance due to uniform nano-filler dispersion
- Easy to shape, yet stays in place
- Excellent radiopacity (>300% AI)


GC Nano-filler Technology Comparison

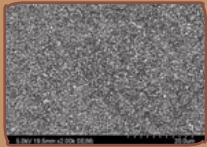

G-aenial Sculpt
(300nm Barium glass)

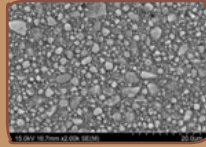

Filtek™ Supreme Ultra*
(4 to 1,000nm Zirconia & Silica)
*Not a registered trademark of GC America.

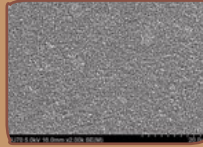

G-aenial Universal Flo
(200nm Strontium glass)

The unique surface treatment of nano inorganic filler and high density & uniform dispersion technology. The technology achieves wear resistance and good physical properties (shining surface texture like natural enamel with lasting esthetics). Even in areas that are difficult to polish, filled G-aenial Sculpt obtains gloss with daily brushing.

G-aenial Sculpt is a great companion to G-aenial Universal Flo. Its fine, 300-nanometer barium glass particle allows for high strength aesthetics, excellent polish and high gloss retention. Its low shrinkage stress and ideal handling properties allow for placement in all classes of restorations. The high filler loading (78% wt) and homogenous dispersion of nano-filler maximizes polish retention and wears better than other universal composite restoratives.

Class III Restoration

Courtesy of Dr. (Prof.) Masashi Miyazaki (Nihon University), Japan




Pre-restoration

Post-restoration

Radiopacity

Courtesy of Dr. (Prof.) Masashi Miyazaki (Nihon University), Japan

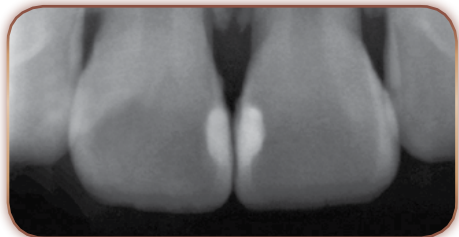

G-aenial Sculpt Syringe Introductory Kit (2.0 mL)

401000	1 syringe of each A1, A2, and A3 and 1 G-Premio BOND 5mL Kit
---------------	---

G-aenial Sculpt Unitip Introductory Kit (0.16 mL each)

401100	20 unitips of each A1, A2, and A3 and 1 G-Premio BOND 5mL Kit
---------------	--

G-aenial Sculpt Syringe Refill (2.0 mL) contains 1 Syringe

009152	A1	009161	AO1
009153	A2	009162	AO2
009154	A3	009163	AO3
009155	A3.5	009164	Bleach White (BW)
009156	A4	009165	Extra Bleach White (XBW)
009157	B1	009166	Cervical (CV)
009158	B2	009167	Juvenile Enamel (JE)
009159	B3	009168	Adult Enamel (AE)
009160	C3		

G-aenial Sculpt 20 Unitip Refill (0.16 mL each) contains 20 tips

009169	A1	009174	B1
009170	A2	009175	B2
009171	A3	009176	B3
009172	A3.5	009177	C3
009173	A4		

G-aenial Sculpt 10 Unitip Refills (0.16 mL each) contains 10 tips

009178	AO1	009182	Extra Bleach White (XBW)
009179	AO2	009183	Cervical (CV)
009180	AO3	009184	Juvenile Enamel (JE)
009181	Bleach White (BW)	009185	Adult Enamel (AE)

