## GC Fuji COAT LC

Prior to use, carefully read  
the instructions for use.

LIGHT CURED PROTECTIVE COATING

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS :

For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24 hours after placement.

### CONTRAINDICATIONS

In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

- Place the glass ionomer cement according to the manufacturer's instructions.
- Using a cotton pellet or brush apply GC Fuji COAT LC to all exposed glass ionomer and adjacent surfaces.
- Light cure for 10 seconds using suitable visible light curing device (470 nm wavelength).
- Close bottle immediately after use.

### STORAGE

Store in a cool and dark place (4-25°C) (39,2-77,0°F).  
(Shelf life : 2 years)

### PACKAGE

Bottle of 5g (5,2mL)

### CAUTION

- GC Fuji COAT LC is FLAMMABLE. Avoid sources of ignition and do not store in direct sunlight.
- In case of contact with oral tissue or skin, flush immediately with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.