Prior to use, carefully read the instructions for use.
**EN**

## GC Fuji II

RADIOPAQUE GLASS IONOMER RESTORATIVE CEMENT

For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

- Restoration of primary teeth.
- Core build-up.
- Restorations of Class III, V and limited Class I cavities.

#### CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

#### DIRECTIONS FOR USE

- POWDER AND LIQUID DISPENSING
  - The standard powder to liquid ratio is 2.7/1.0g,1 level scoop of powder to 1 drop of liquid.
  - For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
  - Hold the liquid bottle vertically and squeeze gently.
  - Close bottles immediately after use.
- MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Mix the first portion with all of the liquid for 10 seconds. Incorporate the remaining portion and mix the whole thoroughly for 15-20 seconds (total time within 30 seconds);
- RESTORATIVE TECHNIQUE
  - Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping use calcium hydroxide.
  - Wash and dry but DO NOT DESICCATE. GC CAVITY CONDITIONER or GC DENTIN CONDITIONER is recommended to remove the smear layer.
  - Mix the required amount of cement. Working time is 1 minute 45 seconds at 23°C (73.4°F). Higher temperatures will shorten working time.
  - Remove surface moisture, but DO NOT DESICCATE.
  - Transfer cement to the preparation using a syringe or suitable placement instrument. Avoid air bubbles.
  - Form the contour, and place a matrix if required during the first 2 minutes of setting
  - Setting time is 5’30” from start of mixing. Once set, remove the matrix and immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure).
- FINISHING
  - One visit treatment
    - After initial set, correct the contour WITHOUT WATER SPRAY using standard techniques.
    - Apply GC Fuji VARNISH or GC Fuji COAT LC and wait for about 15 minutes.
    - Final finishing and polishing under water spray can now be carried out.
    - Apply a final coat of GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.) or G-COAT PLUS (light cure for 20 sec.).
  - Standard method
    - 24 hours after placement finish and polish the restoration under water spray using standard techniques.

#### STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life : 3 years from date of manufacture).

#### SHADE

No. 21 - pale yellow
No. 22 - yellow brown
No. 23 - dark grey
No. 26 - deep brown
M - Modifier


#### PACKAGES

- Bottle of 15g powder with measure.
- Bottle of 10g liquid (8.0 mL).
- 1-1 pkg. : 15g powder, 10g (8.0mL) liquid, powder measure, mixing pad (No. 22), ISO 9917 (Dental water based cements)

#### CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT mix powder or liquid with any other glass ionomer components.

Last revised : 12/2008

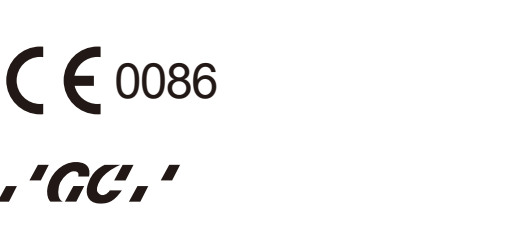MANUFACTURED BY  
**GC CORPORATION**  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY  
**GC CORPORATION**  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

**EU:** GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

**GC AMERICA INC.**  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900

**GC ASIA DENTAL PTE. LTD.**  
19 Loyang Way, #06-27 Singapore 508724  
TEL: +65 6545 7588

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