GC Fuji II

Prior to use, carefully read the instructions for use.

RADIOPAQUE GLASS IONOMER RESTORATIVE CEMENT

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Restoration of primary teeth.
- Core build-up.
- Restorations of Class III, V and limited Class I cavities.

CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

1. POWDER AND LIQUID DISPENSING

- The standard powder to liquid ratio is 2.7/1.0g.1 level scoop of powder to 1 drop of liquid.
- For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- Hold the liquid bottle vertically and squeeze gently.
- Close bottles immediately after use.

2. MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Mix the first portion with all of the liquid for 10 seconds. Incorporate the remaining portion and mix the whole thoroughly for 15-20 seconds (total time within 30 seconds);

3. RESTORATIVE TECHNIQUE

- Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping use calcium hydroxide.
- Wash and dry but DO NOT DESICCATE. GC CAVITY CONDITIONER or GC DENTIN CONDITIONER is recommended to remove the smear layer.
- Mix the required amount of cement. Working time is 1 minute 45 seconds at 23°C (73.4°F). Higher temperatures will shorten working time.
- Remove surface moisture, but DO NOT DESICCATE.
- Transfer cement to the preparation using a syringe or suitable placement instrument. Avoid air bubbles.
- Form the contour, and place a matrix if required during the first 2 minutes of setting.
- Setting time is 5'30" from start of mixing. Once set, remove the matrix and immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (Light Cure).

4. FINISHING

- One visit treatment
 - After initial set, correct the contour WITHOUT WATER SPRAY using standard techniques.
 - Apply GC Fuji VARNISH or GC Fuji COAT LC and wait for about 15 minutes.
 - Final finishing and polishing under water spray can now be carried out.
 - Apply a final coat of GC Fuji VARNISH or GC Fuji COAT LC.
- Standard method
 - 24 hours after placement finish and polish the restoration under water spray using standard techniques.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life : 3 years)

SHADE

No. 21 - pale yellow
No. 22 - yellow brown
No. 23 - dark grey
No. 26 - deep brown
M - Modifier

PACKAGES

- Bottle of 15g powder with measure.
- Bottle of 10g liquid (8.0 mL).
- 1-1 pkg. : 15g powder, 10g (8.0mL) liquid, powder measure, mixing pad (No. 22).
- 3-2 pkg. : 3-15g powder, 2-10g liquid and accessories. ISO 9917 : 1991 (E) (Dental water based cements)

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT mix powder or liquid with any other glass ionomer components.

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