Prior to use, carefully read the instructions for use.

EN

GC Fuji® II LC (Improved)

RADIOPAQUE LIGHT CURED REINFORCED GLASS IONOMER RESTORATIVE

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Class III and V restorations, particularly for cervical erosions and root surface caries.
- Restoration of primary teeth.
- Core build up.
- Cases where radiopacity is required.
- Geriatric applications.
- As a base or liner.

CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

Powder / Liquid Ratio (g/g)	3,2 / 1,0
Mixing Time (sec.)	20"-25"
Working Time (min., sec.)	3'45"
Light Curing Time (sec.)	20"
Depth of Cure (A2) (mm)	1,8

Test conditions: Temperature (23+ / - 1°C)

Relative humidity (50+ / -10%)

- POWDER AND LIQUID DISPENSING**
 - Select shade based on Vita® shade guide.
© : Vita is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.
 - The standard powder to liquid ratio is 3.2g / 1.0g. 1 level scoop of powder to 2 drops of liquid.
 - For accurate dispensing of powder, tap the bottle gently. Do not shake or invert.
 - Hold the liquid bottle vertically and squeeze gently.
 - Close bottles immediately after use.
- MIXING**
 - Fluff powder of bottle before dispensing (Fig. 1).
 - Place one scoop of powder and two drops of liquid on pad. Divide powder in half (Fig. 2).
 - Spread liquid out into a thin layer (about the size of a half dollar or 3 cm) with plastic spatula (Fig. 3).
 - Put half of the powder onto liquid and mix with lapping strokes (like impression material) for 10 to 15 seconds (Fig. 4).
 - Pull in remaining powder and mix thoroughly to a glossy consistency. Do not exceed 20-25 second total mixing time (Fig. 5).
- RESTORATIVE TECHNIQUE**
 - Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping use calcium hydroxide.
 - Apply GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge.
 - Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe. DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening).
 - Mix the required amount of cement. Working time is 3 minutes 45 seconds from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
 - Transfer cement to the preparation using a syringe or suitable placement instrument. Avoid air bubbles.
 - Form the contour and place a transparent matrix if required.
 - Light cure for 20 seconds using a visible light curing device (470 nm wavelength). Place light source as closely as possible to the cement surface.
Note :
For cavities deeper than 1.8 mm, use a layering technique.
- FINISHING**
 - Remove matrix and finish under water spray using standard techniques.
Note :
Apply a final coat of GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.) or G-COAT PLUS (light cure for 20 sec.).

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39.2-77.0°F). (Shelf life : 3 years from date of manufacture.)

SHADE

(11 shades) A1, A2, A3, A3.5, A4, B2, B3, B4, C2, C4, D2.

PACKAGES

- 1-1 package : 15g powder, 8g (6.8mL) liquid, powder scoop, mixing pad (No.22), plastic spatula.
- Bottle of 15g powder with scoop.
- Bottle of 8g (6.8mL) liquid.

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Do not mix powder or liquid with other glass ionomer components.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
<http://www.gceurope.com>
or for The Americas
<http://www.gcamerica.com>
They can also be obtained from your supplier.

CLEANING AND DISINFECTING:
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSER. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised : 03/2016

RxOnly

CE 0086

GC

MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
EU: GC EUROPE N.V.
Researchpark Haasrode-Luven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 10 0

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
11 Tampines Concourse, #03-05, Singapore 528729
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Banksmeadow, NSW 2019, Australia
TEL: +61 2 9316 4499

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