GC Fuji VARNISH

PROTECTIVE COATING

Prior to use, carefully read the instructions for use.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24 hours after placement.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

- Place the glass ionomer cement according to the manufacturer's instructions.
- Using a cotton pellet or brush apply GC Fuji VARNISH to all exposed glass ionomer and adjacent surfaces.
- Dry by gently blowing with air syringe.
- Maintain moisture isolation for 2-3 minutes.
Note :
GC Fuji VARNISH is volatile. Close bottle immediately after use.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).
(Shelf life: 3 years from date of manufacture)

PACKAGE

Bottle of 10g (10.4mL)

CAUTION

- GC Fuji VARNISH is FLAMMABLE. Avoid sources of ignition and do not store in direct sunlight.
- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.