Prior to use, carefully read the instructions for use.

GC PLIERS

PLIERS FOR REMOVAL OF PROSTHETIC APPLIANCES

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. Removal of luted temporary prosthetic appliances.
2. For removal of all types of prosthetic appliances during trial fitting.

DIRECTIONS FOR USE

1. REMOVAL OF TEMPORARY PROSTHETIC APPLIANCES

- a) Use the GC PLIERS without the rubber pads.
- b) Hold the cervical area of the crown with the tips of the pliers.
- c) Twist the crown alternatively towards the mesial and distal direction.
- d) Remove the crown in a labiolingual direction.

2. REMOVAL OF FINAL PROSTHETIC APPLIANCES DURING TRIAL FITTING

- a) Use the GC PLIERS with the rubber pads by pushing the pads on the tips of the pliers.
- b) Place emery powder on the surface of the pads in order to improve the grip.
- c) Continue with 1 b).

CLEANING

Cleaning of the GC Pliers made out of stainless steel, can be accomplished by hand scrubbing, ultrasonic cleaning or thermodesinfection with a non-corrosive, neutral detergent. For further details consult the respective manufacturer's instructions.

STERILISATION

GC Pliers may be routinely sterilized by the following methods : autoclave, chemiclave or dry heat.

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.