

## GC Pliers

### Pliers for Removal of Prosthetic Appliances

For use only by a dental practitioner in the recommended indications.

#### Recommended Indications

Removal of luted temporary prosthetic appliances

For removal of all types of prosthetic appliances during trial fitting

#### **Directions for Use**

##### REMOVAL OF TEMPORARY PROSTHETIC APPLIANCES

Use the GC PLIERS without the rubber pads

Hold the cervical area of the crown with the tips of the pliers

Twist the crown alternatively towards the mesial and distal direction

Remove the crown in a labolingual direction

##### REMOVAL OF FINAL PROSTHETIC APPLIANCES DURING TRIAL FITTING

Use the GC PLIERS with the rubber pads by pushing the pads on the tips of the pliers

Place emery powder on the surface of the pads in order to improve the grip

Continue with 1 b.

#### Cleaning

Cleaning of the GC PLIERS made out of the stainless steel, can be accomplished by hand scrubbing, ultrasonic cleaning or thermodesinfection with a non-corrosive, neutral detergent. For further details consult the respective manufacturer's instructions.

#### Sterilization

GC PLIERS may be routinely sterilized by the following methods: autoclave, chemiclave or dry heat.

#### Packages

Introductory Set: one pair of pliers, 20 rubber pads, 4g. emery powder

Emery Powder, 4 g

Rubber Pads, 50 pieces