## GC TEMP ADVANTAGE®

EUGENOL FREE TEMPORARY CEMENT WITH FLUORIDE,  
POTASSIUM NITRATE AND CHLORHEXIDINE

GB

**Prior to use, carefully read the instructions for use.**

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATION:

Cementation of temporary crowns, bridges, inlays and onlays. This product is not intended to remain in the mouth for more than 30 days.

### CONTRAINDICATIONS:

1. Pulp capping
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### PRIOR TO USE:

Prior to use, assure syringes have conditioned to normal use temperature (68-77°F/20-25°C) for a minimum of 2 hours.

### DIRECTIONS FOR USE (with mixing tip):

1. Complete tooth preparation & fabrication of the temporary restoration prior to extruding cement.
2. Remove the protective cap and discard.
3. Before dispensing the first material from a new cartridge, bleed the pastes in order to prevent the incorporation of air bubbles in the material.
4. Align the "V" shaped notch on the syringe barrel locking tabs with the small tab on the mixing tip. Push the tip firmly to attach the mixing tip. Rotate the tip 1/4 turn clockwise.
5. Extrude a thin layer of cement evenly over the dried inside surface of the restoration.
6. Firmly seat the restoration.
7. This cement will set faster in the mouth than at room temperature (68-77°F/20-25°C). Minimum working time is 2 minutes from the start of the mix. Clean up can begin at 2-4 minutes post insertion. Check mixed material for desired consistency before starting cleanup. Check excess paste with a probe to assure set. When the cement has set, remove excess material with a probe.
8. After use, leave the mixing tip in place. Subsequent uses should not require bleeding the syringe.

### ALTERNATIVE DIRECTIONS FOR USE – Hand Mixing:

Although Temp Advantage is designed to be used with an auto-mix tip it can be hand mixed if desired.

1. Complete tooth preparation and fabrication of the temporary restoration prior to extruding cement.
2. Remove and save the protective cap.
3. Before dispensing the first material from a new cartridge, bleed the pastes in order to prevent the incorporation of air bubbles in the material.
4. Extrude the desired amount of base and accelerator onto a clean mixing pad finishing by wiping the tips along the mixing pad. Replace and lock the protective cap on the syringe. Avoid mixing the base and accelerator when replacing the cap.
5. Using a clean plastic instrument mix the two components together to achieve a uniform, streak free mixture. **Note:** Working time from the start of mixing is 2:00 minutes.
6. Using the instrument place the desired amount of mixed cement evenly over the dried inside surface of the restoration.
7. Firmly seat the restoration.
8. This cement will set faster in the mouth than at room temperature (68-77°F/20-25°C). Minimum working time is 2 minutes from the start of the mix. Clean up can begin at 2-4 minutes post insertion. Check mixed material for desired consistency before starting cleanup. Check excess paste with a probe to assure set. When the cement has set, remove excess material with a probe.

### STORAGE:

Store at normal room temperature (68-77°F/20-25°C). (Shelf life: 2 years from date of manufacture).

### PACKAGES:

Introductory Package: 1 Syringe (7.5g) and 15 mixing tips.

Bulk Package: 4 Syringes (7.5g each)

### CAUTION:

1. In case of contact with oral tissue or skin, remove and flush with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.

Manufactured by:  
GC America Inc.  
3737 W. 127th St.  
Alsip, Illinois 60803

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