

GC TEMP ADVANTAGE™

Eugenol Free
Temporary Cement



- 136505 GC TEMP ADVANTAGE Package**
Contains: One TEMP ADVANTAGE 7.5g syringe and 15 mixing tips.
- 136506 Replacement Automix Tips, 15 per package**
- 136507 GC TEMP ADVANTAGE Bulk Pack**
Contains: Four TEMP ADVANTAGE 7.5g syringes (No Tips)

Related Products

- 000236 K.Y. Crown Removal Pliers Starter Package**
Contains: One K.Y. Crown Removal Plier, 20 rubber tips and emery powder.
- 000237 Replacement Rubber Tips, 50 per package**
- 000235 Emery Powder Refill, 2g bag**
- 338006 Unifast LC Introductory Assortment Package**
Contains: One 30g bottle of each shade (A2, A3, B2, B3, C2 and Translucent) of powder, two 15ml bottles of liquid, two mixing cups, measures, spatula and brush.
- 340501 Alike Professional Assortment Package**
Contains: One 45g bottle each of shade #59 (B1), #62 (A1), #65 (A2), #67 (A3), #77 (B3) and #81 (A3.5) of powder, one 118ml bottle of liquid and liquid dropper.
- 001842 REVOTEK LC Introductory Set**
Contains: One Putty Stick of REVOTEK LC (Universal Shade B2), one light-proof storage case and one GC Spatula No. 2.



GC America Inc.

Advancing the Art and Science of Dentistry

© 2003 GC America Inc., 3737 West 127th St., Alsip, Illinois 60803

<http://www.gcamerica.com>

Lit. Code 833222-1103



GC TEMP ADVANTAGE™
Eugenol Free Temporary Cement

FLUORIDE
CHLORHEXIDINE

POTASSIUM NITRATE



**Everything You Need
Everything You Want**

FOR PROVISIONAL CEMENTATION

GC TEMP ADVANTAGE™

Eugenol Free Temporary Cement with Fluoride, Potassium Nitrate and Chlorhexidine



Basic Package contains one GC Temp Advantage Syringe and 15 Mixing Tips.

Everything You Need

Everything you need in a temporary cement - superior handling, 2 minutes working time, 2 minutes self-cure, simple clean-up, a good seal, high retention, high compressive strength and virtually no sensitivity - soothing. Low film thickness eliminates post-cementation adjustments.

Everything You Want

GC Temp Advantage gives you everything you want for temporary cementation including fluoride, potassium nitrate and chlorhexidine, in an easy-to-use automix syringe! (If desired, GC Temp Advantage pastes can be extruded onto a mixing pad and hand-mixed.

Easy-to-Use Automix Syringe

No more mixing. No more measuring. No wasteful cartridges or messy tubes.

With its paste-paste automix syringe delivery, GC Temp Advantage is extremely easy-to-use with minimal waste.

Non-Eugenol Cement

Compatible with all provisional materials, GC Temp Advantage will not soften, craze or discolor acrylic resin or polycarbonate temporary restorations. It will not burn or irritate soft tissue.

The big difference is the additives

Fluoride. Potassium nitrate. Chlorhexidine. WOW!

Physical Properties

- TEMP ADVANTAGE**
- Working Time @23°C: 2 minutes
 - Setting Time @ 23°C: Two minutes from placement
 - Total Time (Start of mix to final set): 4 minutes
 - Compressive Strength @ 24 hours: 10-12 MPa
 - Film Thickness: 15µm
 - Paste Ratio: 1.3/1.0

Note: Film thickness, compressive strength tested per ADA Specification 30. Temp Advantage should be conditioned to room temperature for 24 hours before use.

Extremely Easy-to-Use


Place a new automix tip on the syringe and express the material as needed to cement the restoration.



Clean and dry finished prep and temporary restoration.



Apply Temp Advantage with automix syringe.

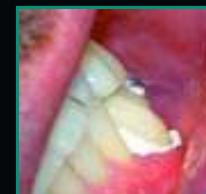

Seat temporary and have patient occlude.



Sets intraorally in under one minute and cleans up easily.



Cemented temporary bridge - superior retention, seal and patient comfort.



NOTE: Fluoride is known to reduce the likelihood of secondary caries - cariostatic. Potassium nitrate has been shown to reduce sensitivity and have a sedative effect on pulp. Chlorhexidine is a proven anti-bacterial agent.