GRADIA® DIRECT LIGHT-CURED COMPOSITE RESTORATIVE

For use only by a dental professional in the recommended indications. RECOMMENDED INDICATIONS

- 1. Direct restorative for Class III, IV, V cavities. 2. Direct restorative for wedge-shaped defects and root surface cavities. 3. Direct restorative for veneers and diastema closure.

- 1. Direct restorative for Class I and II cavities. CONTRAINDICATIONS 1. Pulp capping.

DIRECTIONS FOR USE 1. Shade Selection Clean tooth with pumice and water. Shade selection should be made prior to isolation.

- 1. Basicly, material can be applied in a single layer to achieve aesthetic restorations using Standard shades. 2. Material may be hard to extrude immediately after removing from cold storage.

Clinical Hints 1. Anterior cavities a. In the case of small cavities Restore using a one shade layering technique.

2. Posterior cavities a. Clean case of small cavities Restore using a one shade layering technique.

Examples of Clinical Applications (Clinical Hint No.1, 2) (1-a) Standard (1-b) Standard (1-c) Standard (1-d) Standard (1-e) Standard

Shade combination chart for multiple layers in large cavities of anterior teeth (1-b)

Table with 9 columns (A1-A9, B1-B9, C1-C9) and 4 rows (Inside special, Standard, Outside special, Shade) for anterior teeth.

For details of shades, refer to the following section of SHADES.

2. Contouring before Light Curing Contour using standard techniques.

6. Light cure GRADIA DIRECT using a light curing unit (Fig. 3)

Refer to the following chart for Irradiation Time and Effective Depth of Cure.

GRADIA DIRECT ANTERIOR: Irradiation Time and Effective Depth of Cure

Table with 2 columns (Irradiation time, Effective Depth) and 2 rows (Plasma arc, GC-G-Light) for anterior teeth.

GRADIA DIRECT POSTERIOR: Irradiation Time and Effective Depth of Cure

Table with 2 columns (Irradiation time, Effective Depth) and 2 rows (Plasma arc, GC-G-Light) for posterior teeth.

Note: 1. Material should be placed and light cured in layers. For maximum layer thickness, please consult the tables.

7. Finishing and polishing Finish and polish using diamond burs, polishing points and discs.

Cleaning and Sterilizing of the Unit APPLIER 1. Clean the applicator with gauze soaked in alcohol.

SHADES 1.22 shades for anteriors Standard shade: XBW (Extra Bleaching White), BW (Bleaching White).

PACKAGES 1. Unitips 1. INTRODUCTORY KIT (35 Unitips in 7 shades)

2. REFILL a. Pack of 20 tips (each in 14 shades) (0.16ml, per tip)

3. Option a. Unitip APPLIER b. Shade guide c. Mixing pad (No.14B)

CAUTION 1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol.

11. Do not use GRADIA DIRECT in combination with eugenol containing materials as eugenol may hinder GRADIA DIRECT from setting.

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