# GRADIA® DIRECT X

LIGHT-CURED COMPOSITE RESTORATIVE

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS


- 1. Direct restorative for Class I, II, III, IV, V cavities.
- 2. Direct restorative for wedge-shaped defects and root surface cavities.
- 3. Direct restorative for veneers and diastema closure.

### CONTRAINDICATIONS

- 1. Pulp capping
- 2. Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

### DIRECTIONS FOR USE

- 1. Shade Selection  
Clean the tooth with pumice and water. Shade selection should be made prior to isolation. Select the appropriate GRADIA DIRECT X shades by referring to the GDIA DIRECT X shade guide.
- 2. Cavity Preparation  
Prepare cavity using standard techniques.  
Note:  
For pulp capping, use calcium hydroxide.
- 3. Bonding Treatment  
For bonding GRADIA DIRECT X to enamel and / or dentin, use a light-cured bonding system such as G-Premo BOND, G-anial Bond, G-BOND or GC Fuji BOND LC (Fig. 1). Follow manufacturer's instructions.
- 4. Placement of GRADIA DIRECT X  
1) Dispensing from a Unipit  
Insert the GRADIA DIRECT X Unipit into the Unipit APPLIER II or equivalent. Remove the cap and expel the material directly into the prepared cavity. Use steady pressure (Fig. 2).
- 2) Dispensing from a syringe  
Remove syringe cap and dispense material onto a mixing pad. Place the material into the cavity using a suitable placement instrument. After dispensing, screw syringe plunger anticlockwise by a half to fully turn to release residual pressure inside the syringe. Replace cap immediately after use.
- 2. The material can be applied in a single layer to achieve aesthetic restorations using Standard shades. For details, refer to the Clinical Hints.
- 2. The material may be hard to extrude immediately after removing from cold storage. Prior to use, leave to stand for a few minutes at normal room temperature.
- 3. After dispensing, avoid too long exposure to ambient light. Ambient light can shorten the manipulation time.

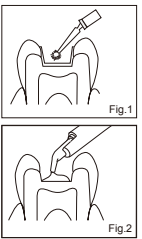### Clinical Hints

- a. In the case of small cavities  
Restore using a one shade technique. In most cases the use of one Standard shade alone will be sufficient. In cases where a higher degree of translucency is needed, one of the Outside Special shades can be selected.
- b. In the case of large and/or deep cavities  
In most cases a multi shade layering technique will give the best aesthetic results. To block out shine through from the oral cavity or to mask discolored dentin, place X-AO2 and continue to build up with a Standard shade. For optimal aesthetics use a Outside special shade as the final composite layer. In the case of deep posterior cavities, a flowable composite such as GRADIA DIRECT Fio /LoFo or a glass ionomer cement such as GC Fuji LINING LC (Paste Pak) or GC Fuji IX GP can be used on the cavity floor instead of an opaque shade. See also Examples of Clinical Applications.

- 5. Contouring before Light Curing  
Contour using standard techniques.
- 6. Light Curing  
Light cure GRADIA DIRECT X using a light curing unit (Fig. 3). Refer to the following chart for irradiation Time and Effective Depth of Cure.

Shade	Irradiation time	High power LED (plus than 1200mW/cm²)	
		10 sec.	20 sec.
X-A1, X-B1, X-WT		20 sec.	40 sec.
X-A2, X-A3, X-B2, X-C2, X-D2		3.0 mm	3.5 mm
X-A3.5, X-AO2, X-XBW, X-BW		2.5 mm	3.0 mm
		2.0 mm	3.0 mm

- Note:
  - 1. The material should be placed and light cured in layers. For maximum layer thickness, please consult the tables.
  - 2. Lower light intensity may cause insufficient curing or discoloration of the material.
- 7. Finishing and Polishing  
Finish and polish using diamond burs, polishing points and discs. To obtain a high gloss, polishing pastes can be used.

### Cleaning and Sterilizing of the Unipit APPLIER II

Please refer to the Manufacturer's Instructions for Use for Cleaning and Sterilization of the Unipit APPLIER II.

### Cleaning of the Shade Guide

The shade guide can be cleaned with water and mild soap. It should not be sterilized. The use of various chemical disinfectants / sterilants may cause damage to the shade guide.

### SHADES

- Standard shade : X-XBW\*, X-BW\*, X-A1, X-A2, X-A3, X-A3.5, X-B1\*, X-B2\*, X-C2\*, X-D2\*
- Inside special shade : X-AO2
- Outside special shade : X-WT\*
- \* Not available in Europe.

### STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39.2-77.0°F) away from high temperatures or direct sunlight.

### PACKAGES

- I. Unipits  
1. Pack of 20 tips (each in 5 shades) (0.16ml per tip)  
X-A1, X-A2, X-A3, X-BW, X-A1, X-A2, X-A3, X-A3.5, X-B1\*, X-B2\*, X-C2\*, X-D2\*
- II. Syringes  
1. 1 syringe (in 12 shades) (2.7mL per syringe)  
Note:  
Weight per syringe : 5.0g
- 2. Option  
a. Unipit APPLIER II b. Shade guide c. Mixing pad (No.14B)

### CAUTION

- 1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water.
- 2. In case of contact with eyes, flush immediately with water and seek medical attention.
- 3. Take care to avoid ingestion of the material.
- 4. Prior to starting the filling process, partially expose paste outside of the patient's mouth and away from the patient to ensure paste is flowing properly.
- 5. Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
- 6. For infection control reasons, Unipits are for single use only.
- 7. Wear protective eyeglasses during light curing.
- 8. When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
- 9. Do not mix with other similar products.
- 10. Avoid getting material on clothing.
- 11. In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light curing.
- 12. Do not use GRADIA DIRECT X in combination with eugenol containing materials as eugenol may hinder GRADIA DIRECT X from setting.
- 13. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- 14. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:  
 http://www.gecorp.com  
 or for The Americas  
 http://www.gcamerica.com  
 They can also be obtained from your supplier.

### CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMEDIATELY. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

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