**ISO FUNCTIONAL****REBASING AND RELINING IMPRESSION MATERIAL**

For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**

Because of its unique slow setting characteristics ISO FUNCTIONAL enables production of extremely accurate impressions. This makes it particularly useful as an impression material for rebasing and muscle trimming.

May also be used for general impressions such as dentures, root surfaces and posts.

**CONTRAINDICATIONS**

In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

**DIRECTIONS FOR USE****REBASING**

1. Remove the flange and tissue contact surface from the denture to a depth of 2-3 mm using steel burs or abrasive points.
2. Cover the relieved surface of the denture with ISO FUNCTIONAL which has been softened over a flame.
3. Soften again by passing through hot water at 45°C (113°F). Insert the denture into the mouth and instruct the patient to bite and hold the denture in centric occlusion. Perform muscle trimming movements.
4. The denture should be retained in the mouth for 1 day to allow a truly functional impression to be formed under normal chewing conditions.  
Note :  
Advise patients to avoid hot drinks during this period.
5. The impression formed is used in standard rebasing procedures.

**NOTES**

- 1) ISO FUNCTIONAL has a fast colour, no taste and no smell and is non-irritating to the patient's mouth.
- 2) To avoid ISO FUNCTIONAL becoming too sticky do not overheat.
- 3) If ISO FUNCTIONAL adheres to the denture, let it set hard and remove with laboratory burs and hand instruments. Remove residual traces using solvent.

**STORAGE**

Store at normal room temperature (23°C / 74°F).  
(Shelf life : 3 years)

**PACKAGES**

ISO FUNCTIONAL STICKS 120g (15 pieces).

**CAUTION**

In case of contact with eyes, remove and flush with water.

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.
NIMI CHEMICAL INDUSTRIAL CO., LTD.