GB

COE® METAL IMPRESSION TRAYS - NICKEL PLATED

For use only by a dental professional in the recommended indications.

Prior to use, carefully read the instructions for use.

RECOMMENDED INDICATIONS

To use with impression material for taking dental impressions.

CONTRAINDICATIONS

In rare cases, product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

PRECAUTIONS

- Do not use dry heat sterilization.
- The use of various chemical disinfectants/sterilants may cause discoloration and/or etching of trays and therefore is not recommended.
- An electrolytic reaction may occur if trays are placed in a liquid medium with dissimilar metals, causing damage to the nickel plating.

DIRECTIONS FOR USE

Handling

- COE METAL IMPRESSION TRAYS – NICKEL PLATED are brass with soldered rim and nickel-plated. Modification of the tray should be slight and carefully made with finger pressure. Do not use pliers. Excessive manipulation will crack the plating.

Cleaning

- To clean a COE METAL IMPRESSION TRAY– NICKEL PLATED, remove the bulk of the impression material from the tray.
- Immerse the tray in a solution of COE TRAY CLEANER and warm water for 15-30 min. Stubborn particles of alginate may then be removed by lightly brushing with a soft bristle brush. COE TRAY CLEANER and warm water may be used for removal of wax and compound.

Sterilization

- COE METAL IMPRESSION TRAYS – NICKEL PLATED may be autoclaved at 250°F/121°C for 30 minutes. Trays should be stored dry.

STORAGE

Store trays in a dry location.

PACKAGES

Regular Perforated – Small, Medium, Large Upper and Lower

Regular Solid – Small, Medium, Large Upper and Lower

Pediatric & Extra Long also available

DK

COE® METAL IMPRESSION TRAYS - NICKEL PLATED

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Opbevares på et tørt sted.

PAKNING

Regular Perforeret – Small, Medium, Large Upper og Lower

Regular Solid - Small, Medium, Large Upper og Lower

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3737 West 127th Street, Alsip, IL 60803, U.S.A.
GC ASIA DENTAL PTE. LTD.
PRINTED IN U.S.A.