Prior to use, carefully read the instructions for use.

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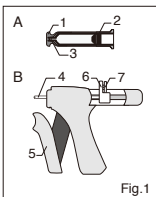
PT

Paste Pak Dispenser II

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

Dispensing material from GC's Paste Pak cartridges

IDENTIFICATION OF PARTS (Fig.1)

- A. Paste Pak Cartridge
1. Cover 2. Inner plunger 3. Cartridge tip
B. Paste Pak Dispenser II
4. Piston 5. Lever 6. Sliding block 7. Piston release

Before use

1. Remove the fixing tape placed between the main part and lever of the dispenser.
2. Remove the V shaped board from the upper side of the dispenser.

DIRECTIONS FOR USE

1. Make sure that the piston is completely retracted into the dispenser (Fig.2) To retract the piston, push the piston release and at the same time pull the sliding block (on the upper side of the dispenser) with fingers. Load the cartridge into the dispenser. Ensure the ▲ mark on the cartridge lines up with the ▲ mark on the dispenser as illustrated. Then turn the cartridge fully to the right position (Fig.3) After cartridge loading, push the piston release forward until it stops (Fig.4.)
 - a) Push the piston release forward.
 - b) Do not push the upper side.

- NOTE:
To remove the cartridge, retract the piston fully and turn the cartridge back to the marked positions.
2. Remove the cartridge cover.
 3. SLOWLY depress the lever to dispense the material.
 4. When the lever is released after dispensing, residual pastes at the cartridge tips are retracted into the cartridge. Clean the cartridge tip and replace the cartridge cover.
- Note:
To avoid damage, do not drop the dispenser.

CLEANING / STERILIZATION

Clean the dispenser with water and soap or detergent. The dispenser may be disinfected using alcohol moistened gauze or steam autoclaved using either of the following conditions.
a. 115-118°C (239-244°F) for 30 minutes
b. 121-124°C (250-255°F) for 15 minutes
c. 126-132°C (259-270°F) for 10 minutes
Use purified water for autoclave. Do not use dry heat sterilization.
The use of various chemical disinfectants / sterilants may cause damage to the dispenser and therefore is not recommended.

- Note:
a. Autoclave temperature should not exceed 132°C (270°F).
b. After autoclave, let the dispenser cool before use.
c. After autoclaving, the release lever of the dispenser may not easily be moved forwards and backwards by fingers. In such a case, slide the release lever forward until it stops. Apply a very small quantity of petroleum jelly onto the entire surface of the piston. Do not apply excess petroleum jelly. Then move the release lever back and forth a few times to ensure smooth movement.

STORAGE

Recommended for optimal performance, store the dispenser away from direct sunlight and away from chemicals such as iodine tincture containing halogen element. Otherwise the dispenser may discolor.

PACKAGE

1 piece

Last revised: 07/2018

'GC'

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

GC SOUTH AMERICA
Rua Heliodora, 399, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESP. TÉC: Mayara de Santis Ribeiro - CRO/SP 105.982

MANUFACTURED by
SHIODA DENTAL CORPORATION
53 Tohbara, Nasu-karasuyama, Tochigi, Japan

RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.

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