Prior to use, carefully read the instructions for use.

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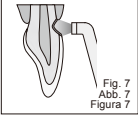
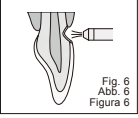
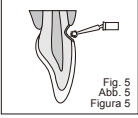
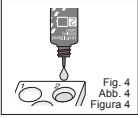
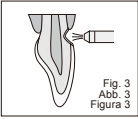
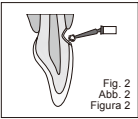
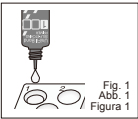
UniFil® Bond

LIGHT-CURED BONDING SYSTEM

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

Bonding of light cured composites to tooth structure.

CONTRAINDICATIONS

1. Pulp capping.
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

1. CAVITY PREPARATION
Prepare tooth in usual manner. Dry the prepared tooth surfaces by gently blowing with an air syringe. Note :
For pulp capping, use calcium hydroxide.

2. APPLICATION OF UniFil® Bond SELF-ETCHING PRIMER
Dispense 1 or 2 drops of UniFil® Bond SELF-ETCHING PRIMER into the dispensing dish provided (Fig. 1). Apply the primer to the enamel and dentin surfaces on the prepared tooth using a pink disposable micro-tip applicator provided (Fig. 2), leave undisturbed for 20 seconds. Note :

- 1) Prior to use, leave the primer bottle at room temperature for 5 minutes after removing from refrigerator.
- 2) After dispensing, use as quickly as possible.
- 3) Do not allow the primer to contact oral tissues. In case of contact, immediately remove with a sponge or cotton. Rinse thoroughly with water after placing composite.
- 4) Replace bottle cap immediately after use.

3. DRYING THE SELF-ETCHING PRIMER
20 seconds after application, gently dry with an air syringe for 5 seconds (Fig. 3) and ensure that the primed surface has a glossy appearance. Do not rinse away. Note :

- Should the applied Primer be rinsed away, dry the cavity and re-apply primer.

4. APPLICATION OF UniFil® Bond BONDING AGENT
Dispense 1 or 2 drops of UniFil® Bond BONDING AGENT into the dispensing dish (Fig. 4). Immediately apply to the primed enamel and dentin surfaces using a yellow colored micro-tip applicator (Fig. 5). Blow gently with an air syringe to form a thin film (Fig. 6).

- Note :
- 1) Prior to use, leave the bottle of BONDING AGENT at room temperature for 5 minutes after removing from refrigerator.

- 2) Take care not to mix with SELF-ETCHING PRIMER.
- 3) After dispensing, protect from light and use as quickly as possible.
- 4) Replace bottle cap immediately after use.
- 5) Prior to blowing gentle air, make sure that no water or oil is injected from the air syringe as this can adversely affect bonding effectiveness.
- 6) Be sure to use UniFil Bond Self-etching Primer with Bonding Agent. Do not use other primers.

5. LIGHT CURING OF BONDING AGENT
Light cure for 10 seconds using a visible light curing unit (470nm wavelength)* (Fig. 7). In case of deep cavities, light cure for a minimum of 20 seconds.

*Irradiation time by light curing device

Halogen	10 sec.
Plasma arc	3 sec.

Note :
When light curing material, wear eye protection glasses.

6. PLACEMENT OF LIGHT-CURED COMPOSITE
For placement of light-cured composite, follow manufacturer's instructions for use.

STORAGE

Store in a refrigerator (2 - 8°C / 35.6 - 46.4°F) when not in use.
(Shelf life : 2 years from date of manufacture)

PACKAGES

1. Kit :

Self-Etching Primer 6mL, Bonding Agent 6mL, 50 micro-tip applicators (25 each pink / yellow), applicator holder, dispensing dish (No. 1) and protective cover (No. 1)

2. Replacement packages

a. Self-Etching Primer 6mL
b. Bonding Agent 6mL

Option :
Box of Micro-tip applicator (250 each pink / yellow) with applicator holder (4)

Other items are available separately.

CAUTION

1. UniFil® Bond SELF-ETCHING PRIMER is flammable. Do not use near naked flame. Keep away from sources of ignition. Keep away from sunlight. Use in a well ventilated place.
2. In case of contact of UniFil® Bond SELF-ETCHING PRIMER and BONDING AGENT with oral tissues or skin, immediately remove with a sponge or cotton. Flush with water. Note that the primer affected part may turn white. In this situation, instruct the patient to leave undisturbed until the mark disappears in 2-3 days.
3. In case of contact with eyes, flush immediately with water and seek medical attention.
4. Take special care that the patient should not swallow the primer or bonding agent.
5. Do not mix with other similar products.
6. Do not use with chemically cured composite resins

Last revised : 04/2009

20086

MANUFACTURED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL: +65 6545 7588

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